LACTATE DEHYDROGENASE-P

Cat. No.	Pack Name	Packaging (Content)
BLT00037	LDH 100	R1: 4 × 20 mL, R2: 1 × 20 mL instruction for use

INTENDED USE


The kit is intended for *in vitro* photometric quantitative determination of lactate dehydrogenase in human serum and plasma on various automatic systems. In combination with other parameters it is intended for screening, monitoring and diagnosis of liver diseases, myocardial infarction, renal damage, anemias. For professional use in clinical laboratories only.

CLINICAL SIGNIFICANCE

Lactate dehydrogenase (LDH) is an enzyme, consisting of five different isoenzymes which catalyze the interconversion of L-lactate and pyruvate. LDH is present in the cytoplasm of all human tissues with higher concentrations in liver, heart and skeletal muscle, and lower values in erythrocytes, pancreas, kidney and stomach. Elevated serum levels of LDH have been observed in a variety of disease states. The highest levels are seen in patients with megaloblastic anemia, disseminated carcinoma and shock. Moderate increases occur in muscular disorders, nephrotic syndrome and cirrhosis. Mild increases in LDH activity have been reported in cases of myocardial or pulmonary infarction, leukemia, hemolytic anemia and non-viral hepatitis.

PRINCIPLE

This assay follows the recommendations of the Deutsche Gesellschaft für Klinische Chemie (DGKC)^{1,2}. LDH catalyzes the reaction between pyruvate and NADH to form L-lactate and NAD⁺.

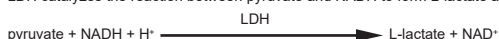The rate of oxidation of NADH is proportional to the activity of LDH present in the sample and decreasing of absorbance can be measured kinetically at 340 nm.

REAGENT DESCRIPTION AND COMPOSITION

R1		R2	
Tris Buffer (pH 7.5)	100 mmol/L	NADH	1.66 mmol/L
Pyruvate	2.0 mmol/L		

COMPOSITION OF REACTION MIXTURE

Tris Buffer (pH 7.5)	78 mmol/L
Pyruvate	1.6 mmol/L
NADH	0.33 mmol/L

REAGENT PREPARATION

Reagents are liquid, ready to use. For monoreagent method, prepare working reagent by mixing of 4 portions of reagent R1 with 1 portion of reagent R2.

MATERIAL REQUIRED BUT NOT PROVIDED WITH THE DEVICE

Any instrument with temperature control of 37 ±0.5 °C that is capable of reading absorbance at 340 nm may be used, general laboratory equipment.
 XL MULTICAL 4×3, Cat. No. XSYS0034
 XL MULTICAL 10×3, Cat. No. XSYS0122
 ERBA NORM 4×5, Cat. No. BLT00080
 ERBA NORM 10×5, Cat. No. XSYS0123
 ERBA PATH 4×5, Cat. No. BLT00081
 ERBA PATH 10×5, Cat. No. XSYS0124

STABILITY AND STORAGE

The unopened reagents are stable till the expiry date stated on the bottle and kit label when stored at 2–8 °C.

Two reagents method – substrate start

Reagents are ready to use. After opening, reagents are stable until expiry date at 2–8 °C if stored at appropriate conditions, closed carefully, protected from light and without any contamination.

Monoreagent method – sample start

Stability of working reagent: 24 hours at 15–25 °C in dark
 5 days at 2–8 °C in dark

SPECIMEN COLLECTION AND HANDLING

It is recommended to follow ISO 15189 and laboratory instruction.

For specimen collection and preparation only use suitable tubes or collection containers.

Only the specimens listed below were tested and found acceptable.

Serum (free from hemolysis).

Plasma: Li-heparin plasma (free from hemolysis).

Plasma may be contaminated with platelets which contain high concentrations of lactate dehydrogenase and should be avoided^{7,8}.

The sample types listed were tested with a selection of sample collection tubes that were commercially available at the time of testing, i.e. not all available tubes of all manufacturers were tested. Sample collection systems from various manufacturers may contain differing materials which could affect the test results in some cases. When processing samples in primary tubes (sample collection systems), follow the instructions of the tube manufacturer.

Centrifuge samples containing precipitates before performing the assay.

See the Limitations and Interferences section for details about possible sample interferences.

Stability in serum / plasma ⁹ :	7 days at	15–25 °C
	4 days at	2–8 °C
	6 weeks at	-20 °C

In connection with certain diseases (e.g. hepatopathy, diseases of skeletal muscle, malignant tumors), the LDH-4 and LDH-5 isoenzyme portions are increased and unstable in cooled and frozen samples; this may lead to an incorrect LDH value in samples collected from patients suffering from such diseases.

Discard contaminated specimens.

CALIBRATION

Calibration with calibrator XL MULTICAL is recommended.

2 point calibration (blank and calibrator); distilled water is recommended as blank

Calibration frequency: it is recommended to do a calibration

- after reagent lot change
- as required by internal quality control procedures

QUALITY CONTROL

For quality control ERBA NORM and ERBA PATH are recommended. The control intervals and limits should be adapted according to each individual laboratory's requirements. Values obtained should fall within the defined intervals. Each laboratory should establish corrective measures to be taken if values fall outside the defined limits.

TRACEABILITY

This method, calibrator XL MULTICAL and controls ERBA NORM and ERBA PATH have been standardized to DGKC recommended method^{1,2}.

ASSAY PROCEDURE

Wavelength: 340 nm

Cuvette: 1 cm

Two reagents method – substrate start

	Reagent blank	Calibrator	Sample
Reagent 1	0.800 mL	0.800 mL	0.800 mL
Sample	–	–	0.020 mL
Calibrator	–	0.020 mL	–
Distilled water	0.020 mL	–	–

Mix and after 1 min. incubation (at 37 °C) add:

Reagent 2	0.200 mL	0.200 mL	0.200 mL
-----------	----------	----------	----------

Mix, incubate 1 min. at 37 °C and then measure the initial absorbance of calibrator and sample against reagent blank. Measure the absorbance change exactly after 1, 2 and 3 min. Calculate 1 minute absorbance change (ΔA/min).

Monoreagent method – sample start

	Reagent blank	Calibrator	Sample
Working reagent	1.000 mL	1.000 mL	1.000 mL
Sample	–	–	0.020 mL
Calibrator	–	0.020 mL	–
Distilled water	0.020 mL	–	–

Mix, incubate 1 min. at 37 °C and then measure the initial absorbance of calibrator and sample against reagent blank. Measure the absorbance change exactly after 1, 2 and 3 min. Calculate 1 minute absorbance change (ΔA/min).

CALCULATION

$$1. \text{LDH (U/L)} = \frac{\Delta A_{\text{sam}} / \text{min.}}{\Delta A_{\text{cal}} / \text{min.}} \times C_{\text{cal}} \quad C_{\text{cal}} = \text{calibrator concentration}$$

$$2. \text{Using factor (f):} \quad \text{LDH (U/L)} = f \times \Delta A / \text{min} \quad f = 8095 \text{ (at 340 nm)}$$

ASSAY PARAMETERS FOR PHOTOMETERS

Mode	Kinetic	Reaction direction	Decreasing
Wavelength (nm)	340	Normal Low U/L	230
Sample Volume (μL)	10/20	Normal High U/L	460
Working Reagent Volume (μL)	500/1000	Linearity Low U/L	30.3
Lag time (sec.)	60	Linearity High U/L	2320
Kinetic interval (sec.)	60	Blank with	Water
No. of readings	3	Absorbance limit (min.)	0.8
Kinetic factor	8095	Units	U/L
Reaction temperature (°C)	37		

UNIT CONVERSION

U/L × 0.0167 = μkat/L

EXPECTED VALUES¹⁰

At 37 °C 230–460 U/L

It is recommended that each laboratory verifies this range or derives reference interval for the population it serves.

ANALYTICAL PERFORMANCE

Data contained within this section is representative for performance on ERBA XL-640 automatic system. Data obtained in your laboratory may differ from these values.

Limit of quantification:

30.3 U/L
 Limit of quantification represents the lowest measurable analyte level. It is calculated as the determined activity of diluted sample to have CV <20 % (n = 30).

Linearity:

2320 U/L
 Linearity is the highest measured activity with recovery within ±10 % from theoretical value.

Precision:

Precision was determined by using controls in an internal protocol with repeatability (n = 20) and intermediate precision (2 aliquots per run, 2 run per day, 20 days). The following results were obtained:

Repeatability	Mean (U/L)	SD (U/L)	CV (%)
Sample 1	343.9	3.81	1.11
Sample 2	679.1	8.00	1.18

Intermediate precision	Mean (U/L)	SD (U/L)	CV (%)
Sample 1	342.2	6.50	1.90
Sample 2	675.4	11.61	1.72

Accuracy

Two different validated control materials were used. Determined bias is -7.6 % at the target value 495.0 U/L and 10.3 % at the target value 840.9 U/L.

Comparison

A comparison between XL-640 automatic system LDH (y) and a commercially available test (x) using 48 samples gave following results:

Linear regression:

$$y = 0.953x - 9.43 \text{ U/L} \quad r = 0.985$$

Passing-Bablok¹¹:

$$y = 0.961x - 5.75 \text{ U/L} \quad r = 0.958$$

Interferences

Criterion: Recovery within ±10 % of initial value of LDH activity in the sample without interfering substance.

Following substances do not interfere: haemoglobin up to 1 g/L, bilirubin up to 40 mg/dL, triglycerides up to 850 mg/dL. Significant hemolysis may increase LDH concentration because of high levels of LDH in the erythrocytes.

Drugs: No interference was found at therapeutic concentrations using common drug panels¹².

Limitations:

- Deteriorated reagents (e.g. exceeding the storage temperature) may give incorrect results. Minimal allowable absorbance of the reagent blank measured at 340 nm against the distilled water is 0.8.

- High concentration of haemoglobin, bilirubin and triglycerides in sample can interfere with determination of LDH. See paragraph Interferences.

WARNING AND PRECAUTIONS

For *in vitro* diagnostic use. To be handled by entitled and professionally educated person.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

Hazards identification in accordance with Regulation (EC) No 1272/2008

R1, R2

Reagents of the kit are not classified as dangerous.

WASTE MANAGEMENT

Please refer to local legal requirements.

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