# URIC ACID

| Cat. No. | Pack Name | Packaging (Content)   |
|----------|-----------|---|
| XSYS0021 | UA 275    | R1: 5 × 44 mL, R2: 5 × 11 mL, RFID tag, instruction for use |

## INTENDED USE

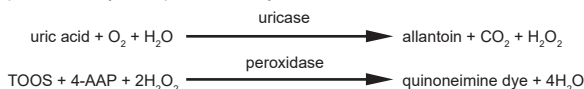
The kit is intended for *in vitro* photometric quantitative determination of uric acid in human serum, plasma and urine on automatic systems ERBA XL. In combination with other parameters it is intended for screening, monitoring and diagnosis of renal dysfunction, gout, leukemia, polycythaemia, atherosclerosis, diabetes and hypothyroidism. For professional use in clinical laboratories only.

## CLINICAL SIGNIFICANCE

Uric acid is a metabolite of purines, nucleic acids and nucleoproteins, consequently, abnormal levels may be indicative of a disorder in the metabolism of these substances. Hyperuricaemia may be observed in renal dysfunction, gout, leukemia, polycythaemia, atherosclerosis, diabetes and hypothyroidism. Decreased levels are present in patients with Wilson's Disease.

## PRINCIPLE

Determination of uric acid is based on modified Trinder's reaction<sup>1</sup>. Uric acid is oxidized to allantoin by uricase with the production of hydrogen peroxide (H<sub>2</sub>O<sub>2</sub>). The H<sub>2</sub>O<sub>2</sub> reacts with 4-aminoantipyrine (4-AAP) and N-ethyl-N-(2-hydroxy-3-sulfo-propyl)-3-methylaniline (TOOS) in the presence of peroxidase to yield a quinoneimine dye<sup>2,3,4,5</sup>:

The resulting change in absorbance at 546 nm is proportional to the uric acid concentration in the sample.

## REAGENT DESCRIPTION AND COMPOSITION

| R1                    |              | R2                    |             |
|-----------------------|--------------|-----------------------|-------------|
| Pipes Buffer (pH 7.0) | 50 mmol/L    | Pipes Buffer (pH 7.0) | 50 mmol/L   |
| 4-AAP                 | 0.375 mmol/L | TOOS                  | 1.92 mmol/L |
| Uricase               | ≥0.2 KU/L    | Peroxidase            | ≥5 KU/L     |
| Sodium azide          | 0.5 g/L      | Sodium azide          | 0.5 g/L     |

## COMPOSITION OF REACTION MIXTURE

|                       |              |
|-----------------------|--------------|
| Pipes Buffer (pH 7.0) | 49 mmol/L    |
| 4-AAP                 | 0.294 mmol/L |
| TOOS                  | 0.376 mmol/L |
| Uricase               | ≥0.16 mmol/L |
| Peroxidase            | ≥1.0 mmol/L  |
| Sodium azide          | 0.49 g/L     |

## REAGENT PREPARATION

Reagents are liquid, ready to use. Load the number of tests from the RFID tag before using a new kit.

## MATERIAL REQUIRED BUT NOT PROVIDED WITH THE DEVICE

XL MULTICAL 4×3, Cat. No. XSYS0034  
 XL MULTICAL 10×3, Cat. No. XSYS0122  
 ERBA NORM 4×5, Cat. No. BLT00080  
 ERBA NORM 10×5, Cat. No. XSYS0123  
 ERBA PATH 4×5, Cat. No. BLT00081  
 ERBA PATH 10×5, Cat. No. XSYS0124  
 Erba XL analysers: XL-200, Cat. No. INS00002  
 XL-640, Cat. No. INS00008  
 XL-1000, Cat. No. INS00010

## STABILITY AND STORAGE

The unopened reagents are stable till the expiry date stated on the bottle and kit label when stored at 2–8 °C.

On board stability: min. 60 days if refrigerated (2–10 °C) and not contaminated.

## SPECIMEN COLLECTION AND HANDLING

It is recommended to follow ISO 15189 and laboratory instruction.

For specimen collection and preparation only use suitable tubes or collection containers.

Only the specimens listed below were tested and found acceptable.

Serum.

Plasma: Li-heparin and K<sub>2</sub>-EDTA plasma.

Urine: Do not refrigerate. For the determination in urine use 24 hours specimen. To prevent the precipitation of uric acid add 15 mL 5 mol/L NaOH into the urine collector to ensure urine pH >8. Dilute urine samples in 1+9 ratio with distilled water and multiply results by 10.

The sample types listed were tested with a selection of sample collection tubes that were commercially available at the time of testing, i.e. not all available tubes of all manufacturers were tested. Sample collection systems from various manufacturers may contain differing materials which could affect the test results in some cases. When processing samples in primary tubes (sample collection systems), follow the instructions of the tube manufacturer.

Centrifuge samples containing precipitates before performing the assay.

See the Limitations and Interferences section for details about possible sample interferences.

|   |             |          |
|---|-------------|----------|
| <b>Stability in serum / plasma<sup>6</sup>:</b> | 3 days at   | 20–25 °C |
|   | 7 days at   | 4–8 °C   |
|   | 6 months at | -20 °C   |

|   |           |          |
|---|-----------|----------|
| <b>Stability in urine<sup>6</sup> (upon NaOH addition):</b> | 4 days at | 20–25 °C |
|---|-----------|----------|

Discard contaminated specimens.

## CALIBRATION

Calibration with calibrator XL MULTICAL is recommended.

2 point calibration (blank and calibrator); distilled water is recommended as blank

Calibration frequency: 30 days

Calibration is needed:

- after reagent lot change
- as required by internal quality control procedures
- calibration interval may be extended based on acceptable verification of calibration by the laboratory

## QUALITY CONTROL

For quality control ERBA NORM and ERBA PATH are recommended.

The control intervals and limits should be adapted according to each individual laboratory's requirements. Values obtained should fall within the defined intervals. Each laboratory should establish corrective measures to be taken if values fall outside the defined limits.

## TRACEABILITY

This method, calibrator XL MULTICAL and controls ERBA NORM and ERBA PATH have been standardized against to ID/MS<sup>7</sup>.

## ASSAY PROCEDURE AND CALCULATION

ERBA XL automatic systems calculate the concentration of each sample. For assay parameters see www.erba.com.

## Assay parameters for ERBA XL automatic systems

|                           |                           |
|---------------------------|---------------------------|
| Assay type                | 2-Point                   |
| Curve type                | Linear                    |
| Wavelength (prim. / sec.) | 546/700 nm                |
| Reading time 1            | just before adding of R2  |
| Reading time 2            | 10 min after adding of R1 |
| Reaction direction        | Increase                  |
| Unit                      | mg/dL (μmol/L)            |
| Reagent volumes           |                           |
| R1                        | 200 μL                    |
| R2                        | 50 μL                     |
| Sample                    | 5 μL                      |

Note: reagents and sample volumes can be different for individual ERBA XL automatic systems depending on the minimum measured volume in the cuvette. The ratio R1:R2:sample does not change.

## UNIT CONVERSION

mg/dL × 60 = μmol/L

## EXPECTED VALUES<sup>8</sup>

At 37 °C

### Serum

Adults Male: 3.5–7.2 mg/dL      Adults Female: 2.6–6.0 mg/dL

### Urine, 24 h:

Average diet: 250–750 mg/24h

High-prine diet: <1000 mg/24h

Low-purine diet: <480 mg/24h

Purine-free diet: <420 mg/24h

It is recommended that each laboratory verifies this range or derives reference interval for the population it serves.

## ANALYTICAL PERFORMANCE

Data contained within this section is representative for performance on ERBA XL-640 automatic system. Data obtained in your laboratory may differ from these values. Data for other ERBA XL automatic systems are available on www.erba.com.

## Limit of quantification:

Serum / plasma 0.10 mg/dL

Urine 0.43 mg/dL

Limit of quantification represents the lowest measurable analyte level. It is calculated as the determined activity of diluted sample to have CV <20% (n = 30).

## Linearity:

Serum / plasma 30.0 mg/dL

Urine 300 mg/dL

Linearity is the highest measured activity with recovery within ±10% from theoretical value.

## Precision:

Precision was determined by using controls in an internal protocol with repeatability (n = 20) and intermediate precision (2 aliquots per run, 2 run per day, 20 days). The following results were obtained:

| Repeatability (serum) | Mean (mg/dL) | SD (mg/dL) | CV (%) | Intermediate precision (serum) | Mean (mg/dL) | SD (mg/dL) | CV (%) |
|-----------------------|--------------|------------|--------|--------------------------------|--------------|------------|--------|
| Sample 1              | 6.07         | 0.067      | 1.10   | Sample 1                       | 5.95         | 0.151      | 2.55   |
| Sample 2              | 11.56        | 0.058      | 0.50   | Sample 2                       | 11.35        | 0.187      | 1.65   |

| Repeatability (urine) | Mean (mg/dL) | SD (mg/dL) | CV (%) | Intermediate precision (urine) | Mean (mg/dL) | SD (mg/dL) | CV (%) |
|-----------------------|--------------|------------|--------|--------------------------------|--------------|------------|--------|
| Sample 1              | 19.1         | 0.22       | 1.18   | Sample 1                       | 25.7         | 0.94       | 3.67   |
| Sample 2              | 31.0         | 0.22       | 0.70   | Sample 2                       | 49.5         | 2.08       | 4.20   |

## Accuracy

Two different validated control materials for serum and urine were used. Determined bias is 11.8% at the target value 7.61 mg/dL, 11.7% at the target value 6.02 mg/dL for serum, 19.9% at the target value 5.42 mg/dL and 20.7% at the target value 11.63 mg/dL for urine.

## Comparison

A comparison between XL-640 automatic system URIC ACID (y) and a commercially available test (x) using 132 samples (serum) gave following results:

Linear regression:

$$y = 1.078x - 0.110 \text{ mg/dL} \quad r = 0.995$$

Passing-Bablok<sup>9</sup>:

$$y = 1.092x - 0.167 \text{ mg/dL} \quad r = 0.989$$

## Interferences

Criterion: Recovery within ±10% of initial value of uric acid concentration in the sample (serum) without interfering substance.

Following substances do not interfere: haemoglobin up to 12.5 g/L, bilirubin up to 24 mg/dL, triglycerides up to 850 mg/dL.

Drugs:

Serum: No interference was found at therapeutic concentrations using common drug panels except Dicyclone, N-acetylcysteine, Metamizole and Acetaminofen (including metabolite N-acetyl-p-benzoquinone imine)<sup>10,11,12,13,14</sup>.

Urine: No interference was found at therapeutic concentrations using common drug panels<sup>10</sup>.

## Limitations:

- Deteriorated reagents (e.g. exceeding the storage temperature) may give incorrect results. Quality of reagents is monitored on automatic systems ERBA XL by checking of the maximum permissible absorbance value of blank.

- High concentration of haemoglobin, bilirubin and triglycerides in sample can interfere with determination of uric acid. Some drugs can also interfere. See paragraph Interferences.

## WARNING AND PRECAUTIONS

For *in vitro* diagnostic use. To be handled by entitled and professionally educated person.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and / or the patient is established.

## Hazards identification in accordance with Regulation (EC) No 1272/2008

### R1, R2

Reagents of the kit are not classified as dangerous.

## WASTE MANAGEMENT

Please refer to local legal requirements.

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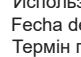
1. Barham D, Trinder P. An improved color reagent for the determination of blood glucose by the oxidase system. *Analyst* 1972; 97: 142-5.
2. Kageyama N. A direct colorimetric determination of uric acid in serum and urine with uricase-catalase system *Clin Chim Acta* 1971;31:421-426. 3. Praetorius E, Poulsen H. Enzymatic determination of uric acid; with detailed directions. *Scand J Clin Lab Invest* 1953;5(3):273-280.
3. Town MH, Gehm S, Hammer B, et al. A sensitive colorimetric method for the enzymatic determination of uric acid. *J Clin Chem Clin Biochem* 1985;23:591.
4. Trivedi R, Rebar L, Berta E, et al. New enzymatic method for serum uric acid at 500 nm. *Clin Chem* 1978; 24(11):1908-11.
5. Kabasakalian P, Kalliney S, Wescott A. Determination of uric acid in serum, with use of uricase and tribromophenol-aminoantipyrene chromogen. *Clin Chem* 1973;19:522.
6. WHO Publication: Use of anticoagulants in diagnostic laboratory investigations, WHO/DIL/LAB/99.1 Rev.2:Jan 2002.
7. Siekmann L. Determination of uric acid in human serum by isotope dilution-mass spectrometry. *J Clin Chem Clin Biochem* 1985;23:129-135.
8. Tietz Textbook of Clinical Chemistry and Molecular diagnostics. Burtis, C.A., Ashwood, E. R., Bruns, D.E.; 5th edition, WB Saunders Company, 2012.
9. Bablok W, Passing H, Bender R, et al. A general regression procedure for method transformation. Application of linear regression procedures for method comparison studies in clinical chemistry, Part III. *J Clin Chem Clin Biochem* 1988 Nov; 26(11): 783-790.
10. Sonntag O, Scholer A. Drug interference in clinical chemistry: recommendation of drugs and their concentrations to be used in drug interference studies. *Ann Clin Biochem* 38, 376-385, 2001.
11. Dastych M, Wiewiorka O, Benovska M. Ethamsylate (Dicynone) Interference in Determination of Serum Creatinine, Uric Acid, Triglycerides, and Cholesterol in Assays Involving the Trinder Reaction; *In Vivo and In Vitro*. *Clin Lab* 60, 1373-1376, 2014.
13. Steinbach D, Racek J, Rajdl D, Interference of natural metabolites and drugs in enzymatic determination of creatinine and uric acid, *Klin. Biochem. Metab.*, 29 (50), 132-138, 2021.
14. Genzen JR, Hunsaker JJ, Nelson LS, Faine BA, Krasowski MD, N-acetylcysteine interference of Trinder-based assays, *Clin Biochem* 49, 100-104, 2016.

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## LITERATÚRA

1. Barham D, Trinder P. An improved color reagent for the determination of blood glucose by the oxidase system. *Analyst* 1972; 97: 142-5.
2. Kageyama N. A direct colorimetric determination of uric acid in serum and urine with uricase-catalase system *Clin Chim Acta* 1971;31:421-426. 3. Praetorius E, Poulsen H. Enzymatic determination of uric acid; with detailed directions. *Scand J Clin Lab Invest* 1953;5(3):273-280.
3. Town MH, Gehm S, Hammer B, et al. A sensitive colorimetric method for the enzymatic determination of uric acid. *J Clin Chem Clin Biochem* 1985;23:591.
4. Trivedi R, Rebar L, Berta E, et al. New enzymatic method for serum uric acid at 500 nm. *Clin Chem* 1978; 24(11):1908-11.
5. Kabasakalian P, Kalliney S, Wescott A. Determination of uric acid in serum, with use of uricase and tribromophenol-aminoantipyrine chromogen. *Clin Chem* 1973;19:522.
6. WHO Publication: Use of anticoagulants in diagnostic laboratory investigations, WHO/DIL/LAB/99.1 Rev.2:Jan 2002.
7. Siekmann L. Determination of uric acid in human serum by isotope dilution-mass spectrometry. *J Clin Chem Clin Biochem* 1985;23:129-135.
8. Tietz Textbook of Clinical Chemistry and Molecular diagnostics. Burtis, C.A., Ashwood, E. R., Bruns, D.E.; 5th edition, WB Saunders Company, 2012.
9. Bablok W, Passing H, Bender R, et al. A general regression procedure for method transformation. Application of linear regression procedures for method comparison studies in clinical chemistry, Part III. *J Clin Chem Clin Biochem* 1988 Nov; 26(11): 783-790.
10. Sonntag O, Scholer A. Drug interference in clinical chemistry: recommendation of drugs and their concentrations to be used in drug interference studies. *Ann Clin Biochem* 38, 376-385, 2001.
11. Dastych M, Wiewiorka O, Benovska M. Ethamsylate (Dicynone) Interference in Determination of Serum Creatinine, Uric Acid, Triglycerides, and Cholesterol in Assays Involving the Trinder Reaction; In Vivo and In Vitro. *Clin Lab* 60, 1373-1376, 2014.
13. Steinbach D, Racek J, Rajdl D, Interference of natural metabolites and drugs in enzymatic determination of creatinine and uric acid, *Klin. Biochem. Metab.*, 29 (50), 132-138, 2021.
14. Genzen JR, Hunsaker JJ, Nelson LS, Faine BA, Krasowski MD, N-acetylcysteine interference of Trinder-based assays, *Clin Biochem* 49, 100-104, 2016.

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