# CREATINE KINASE

Cat. No.	Pack Name	Packaging (Content)
BLT00017	CK 100	R1: 4 × 20 mL, R2: 1 × 20 mL, instruction for use
BLT00086	CK 250	R1: 4 × 50 mL, R2: 1 × 50 mL, instruction for use

## INTENDED USE

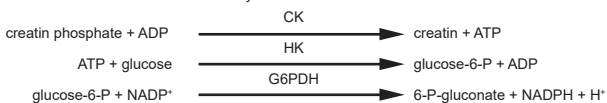
The kit is intended for *in vitro* photometric quantitative determination of creatine kinase in human serum and plasma on various automatic systems. In combination with other parameters it is intended for monitoring and diagnosis of myocardial or skeletal muscle disease. For professional use in clinical laboratories only.

## CLINICAL SIGNIFICANCE

Creatine kinase (CK) is a dimeric enzyme occurring in four different forms: a mitochondrial isoenzyme and the cytosolic isoenzymes CK-MM (muscle type), CK-BB (brain type) and CK-MB (myocardial type). The determination of CK and CK-isoenzyme activities is utilized in the diagnosis and monitoring of myocardial infarction and myopathies such as the progressive Duchenne muscular dystrophy. Following injury to the myocardium, as occurs with acute myocardial infarction, CK is released from the damaged myocardial cells. In early cases a rise in the CK activity can be found just 4 hours after an infarction, the CK-activities reaches a maximum after 12–24 hours and then falls back to the normal range after 3–4 days. Myocardial damage is very likely when the total CK activity is above 190 U/L, the CK-MB activity is above 24 U/L (37 °C) and the CK-MB activity fraction exceeds 6 % of the total.

## PRINCIPLE

The assay method using creatine phosphate and ADP was first described by Oliver<sup>1</sup>, modified by Rosalki<sup>2</sup> and further improved for optimal test conditions by Szasz et al<sup>3</sup>. CK is rapidly inactivated by oxidation of the sulphhydryl groups in the active center. The enzyme can be reactivated by the addition of N-acetylcysteine (NAC)<sup>3</sup>. Interference by adenylate kinase is prevented by the addition of diadenosine pentaphosphate<sup>4</sup> and AMP<sup>3,4</sup>. Standardized methods for the determination of CK using the "reverse reaction" and activation by NAC were recommended by the German Society for Clinical Chemistry (DGKC)<sup>5</sup> in 1977 and the International Federation of Clinical Chemistry (IFCC)<sup>6</sup> in 1991. In 2002 the IFCC confirmed their recommendation and extended it to 37 °C<sup>7</sup>. This assay meets the recommendations of the IFCC and DGKC.

The rate of formation of NADPH is proportional to the activity of CK present in the sample and can be measured kinetically at 340 nm.

## REAGENT DESCRIPTION AND COMPOSITION

R1		R2	
Imidazole buffer, pH 6.1	125 mmol/L	Imidazole buffer, pH 8.9	125 mmol/L
Glucose	25 mmol/L	ADP	15.2 mmol/L
Magnesium acetate	12.5 mmol/L	D-glucose-6-phosphate-dehydrogenase	>8800 U/L
EDTA	2 mmol/L	Creatine phosphate	250 mmol/L
N-acetylcysteine	25 mmol/L	AMP	25 mmol/L
NADP <sup>+</sup>	2.4 mmol/L	Diadenosine pentaphosphate	103 μmol/L
Hexokinase	>6800 U/L		

## COMPOSITION OF REACTION MIXTURE

Imidazole buffer	120 mmol/L	Hexokinase	>5230 U/L
Glucose	19.2 mmol/L	ADP	2.9 mmol/L
Magnesium acetate	9.6 mmol/L	D-glucose-6-phosphate-dehydrogenase	>1692 U/L
EDTA	1.5 mmol/L	Creatine phosphate	48 mmol/L
N-acetylcysteine	19.2 mmol/L	AMP	4.8 mmol/L
NADP <sup>+</sup>	1.8 mmol/L	Diadenosine pentaphosphate	20 μmol/L

## REAGENT PREPARATION

Reagents are liquid, ready to use. For monoreagent method, prepare working reagent by mixing of 4 portions of reagent R1 with 1 portion of reagent R2.

## MATERIAL REQUIRED BUT NOT PROVIDED

Any instrument with temperature control of 37 ± 0.5 °C that is capable of reading absorbance at 340 nm may be used, general laboratory equipment.  
 XL MULTICAL 4×3, Cat. No. XSYS0034  
 XL MULTICAL 10×3, Cat. No. XSYS0122  
 ERBA NORM 4×5, Cat. No. BLT00080  
 ERBA NORM 10×5, Cat. No. XSYS0123  
 ERBA PATH 4×5, Cat. No. BLT00081  
 ERBA PATH 10×5, Cat. No. XSYS0124

## STABILITY AND STORAGE

The unopened reagents are stable till the expiry date stated on the bottle and kit label when stored at 2–8 °C.

### Two reagents method – substrate start

Reagents are ready to use. After the first opening the vials, reagents are stable for 30 days at 2–8 °C if stored at appropriate conditions, closed carefully, protected from light and without any contamination.

### Monoreagent method – sample start

Stability of working reagent: 2 days at 15–25 °C in dark  
 2 weeks at 2–8 °C in dark

## SPECIMEN COLLECTION AND HANDLING

It is recommended to follow ISO 15189 and laboratory instruction.  
 For specimen collection and preparation only use suitable tubes or collection containers.  
 Only the specimens listed below were tested and found acceptable.

Serum.  
 Plasma: Li-heparin and K<sub>2</sub>-, K<sub>3</sub>-EDTA plasma.  
 The sample types listed were tested with a selection of sample collection tubes that were commercially available at the time of testing, i.e. not all available tubes of all manufacturers were tested. Sample collection systems from various manufacturers may contain differing materials which could affect the test results in some cases. When processing samples in primary tubes (sample collection systems), follow the instructions of the tube manufacturer.  
 Centrifuge samples containing precipitates before performing the assay.  
 See the Limitations and Interferences section for details about possible sample interferences.

**Stability in serum / plasma<sup>10</sup>:** 2 days at 15–25 °C  
 7 days at 2–8 °C  
 4 weeks at -20 °C

Discard contaminated specimens.

## CALIBRATION

Calibration with calibrator XL MULTICAL is recommended.  
 2 point calibration (blank and calibrator); distilled water is recommended as blank  
 Calibration frequency: it is recommended to do a calibration  
 • after reagent lot change  
 • as required by internal quality control procedures

## QUALITY CONTROL

For quality control ERBA NORM and ERBA PATH are recommended.  
 The control intervals and limits should be adapted according to each individual laboratory's requirements. Values obtained should fall within the defined intervals. Each laboratory should establish corrective measures to be taken if values fall outside the defined limits.

## TRACEABILITY

This method, calibrator XL MULTICAL and controls ERBA NORM and ERBA PATH have been standardized to IFCC recommended method<sup>6</sup>.

## ASSAY PROCEDURE

Wavelength: 340 nm  
 Cuvette: 1 cm

### Two reagents method – substrate start

	Reagent blank	Calibrator	Sample
Reagent 1	0.800 mL	0.800 mL	0.800 mL
Sample	–	–	0.040 mL
Calibrator	–	0.040 mL	–
Distilled water	0.040 mL	–	–

Mix and after 5 min. incubation (at 37 °C) add:

Reagent 2	0.200 mL	0.200 mL	0.200 mL

Mix, incubate 3 min. at 37 °C and then measure the initial absorbance of calibrator and sample against reagent blank. Measure the absorbance change exactly after 1, 2 and 3 min. Calculate 1 minute absorbance change (ΔA/min).

### Monoreagent method – sample start

	Reagent blank	Calibrator	Sample
Working reagent	1.000 mL	1.000 mL	1.000 mL
Sample	–	–	0.040 mL
Calibrator	–	0.040 mL	–
Distilled water	0.040 mL	–	–

Mix, incubate 3 min. at 37 °C and then measure the initial absorbance of calibrator and sample against reagent blank. Measure the absorbance change exactly after 1, 2 and 3 min. Calculate 1 minute absorbance change (ΔA/min).

## CALCULATION

- CK (U/L) =  $\frac{\Delta A_{\text{sam}}/\text{min.}}{\Delta A_{\text{cal}}/\text{min.}} \times C_{\text{cal}}$   $C_{\text{cal}}$  = calibrator concentration
- Using factor (f):  
 CK (U/L) = f × ΔA/min  $f = 4127$  (at 340 nm)

## ASSAY PARAMETERS FOR PHOTOMETERS

Mode	Kinetic	Reaction direction	Increasing
Wavelength (nm)	340	Normal Low U/L	24
Sample Volume (μL)	40	Normal High U/L	171
Working Reagent Volume (μL)	1000	Linearity Low U/L	5.12
Lag time (sec.)	180	Linearity High U/L	2340
Kinetic interval (sec.)	60	Blank with	Water
No. of readings	3	Absorbance limit (max.)	0.3
Kinetic factor	4127	Units	U/L
Reaction temperature (°C)	37		

## UNIT CONVERSION

U/L × 0.0167 = μkat/L

## EXPECTED VALUES<sup>11</sup>

At 37 °C

Females:	24–145 U/L	Children <sup>12</sup> :	
Males:	46–171 U/L	Umbilical cord blood	175–402 U/L
		Newborns	468–1200 U/L
		≤5 days	195–700 U/L
		<6 months	41–330 U/L
		>6 months	24–229 U/L

It is recommended that each laboratory verifies this range or derives reference interval for the population it serves.

## ANALYTICAL PERFORMANCE

Data contained within this section is representative for performance on ERBA XL-640 automatic system. Data obtained in your laboratory may differ from these values.

**Limit of quantification:** 5.12 U/L

Limit of quantification represents the lowest measurable analyte level. It is calculated as the determined activity of diluted sample to have CV <20% (n = 30).

**Linearity:** 2340 U/L

Linearity is the highest measured activity with recovery within ±10 % from theoretical value.

## Precision:

Precision was determined by using controls in an internal protocol with repeatability (n = 20) and intermediate precision (2 aliquots per run, 2 run per day, 20 days). The following results were obtained:

Repeatability	Mean (U/L)	SD (U/L)	CV (%)	Intermediate precision	Mean (U/L)	SD (U/L)	CV (%)
Sample 1	133.0	0.86	0.64	Sample 1	143.1	2.99	2.09
Sample 2	313.3	0.86	0.27	Sample 2	333.3	6.05	1.81

## Accuracy

Two different validated control materials were used. Determined bias is 2.1 % at the target value 208.1 U/L and -2.5 % at the target value 513.2 U/L.

## Comparison

A comparison between XL-640 automatic system CK (y) and a commercially available test (x) using 51 samples gave following results:

Linear regression:

$$y = 0.975x - 8.47 \text{ U/L} \quad r = 0.991$$

$$\text{Passing-Bablok}^{13}: y = 0.992x - 4.00 \text{ U/L} \quad r = 0.990$$

## Interferences

**Criterion:** Recovery within ±10 % of initial value of CK activity in the sample without interfering substance. Following substances do not interfere: haemoglobin up to 7 g/L, bilirubin up to 40 mg/dL, triglycerides up to 850 mg/dL.

**Drugs:** No interference was found at therapeutic concentrations using common drug panels<sup>14</sup>.

## Limitations:

- Deteriorated reagents (e.g. exceeding the storage temperature) may give incorrect results. Maximum allowable absorbance of the reagent blank measured at 340 nm against the distilled water is 0.3.
- High concentration of haemoglobin, bilirubin and triglycerides in sample can interfere with determination of CK. See paragraph Interferences.

## WARNING AND PRECAUTIONS

For *in vitro* diagnostic use. To be handled by entitled and professionally educated person. Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

## Hazards identification in accordance with Regulation (EC) No 1272/2008

**R1**  
 UFI: JXTU-AWJV-3J5M-K095

**R2**  
 UFI: 81UU-UW88-EJ53-7AV7

**R1, R2**

Danger

Contains: imidazole

## Hazard statement:

H360D May damage the unborn child.

## Precautionary statement:

- P201 Obtain special instructions before use.
- P280 Wear protective gloves / protective clothing / eye protection / face protection.
- P308 + P313 IF exposed or concerned: Get medical advice / attention.

## WASTE MANAGEMENT

Please refer to local legal requirements.

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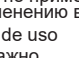
1. Oliver IT. A spectrophotometric method for the determination of creatine phosphokinase and myokinase. *Biochem J* 1955;61:116-122.
2. Rosalki SB. An improved procedure for serum creatine phosphokinase determination. *J Lab Clin Med* 1967;69:696-705.
3. Szasz G, Gruber W, Bernt E. Creatine kinase in serum: 1. Determination of optimum reaction conditions. *Clin Chem* 1976;22(5):650-656.
4. Standard method for the determination of creatine kinase activity. *J Clin Chem Clin Biochem* 1977;15:249-260.
5. Hørder M, Elser RC, Gerhardt M, et al. Approved Recommendation on IFCC Methods for the Measurement of Catalytic Concentration of Enzymes. Part 7. IFCC Method for Creatine Kinase. *Eur J Clin Chem Clin Biochem* 1991;29:435-456.
6. Schumann G, Bonora R, Ceriotti F, et al. IFCC Primary Reference Procedures for the Measurement of Catalytic Activity Concentrations of Enzymes at 37 °C – Part 2. Reference Procedure for the Measurement of Catalytic Concentration of Creatine Kinase. *Clin Chem Lab Med* 2002;40(6):635-642.
7. Klauke R, Schmidt E, Lorentz K. Recommendations for carrying out standard ECCLS procedures (1988) for the catalytic concentrations of creatine kinase, aspartate aminotransferase, alanine aminotransferase and  $\gamma$ -glutamyltransferase at 37 °C. *Eur J Clin Chem Clin Biochem* 1993;31:901-909.
8. Thomas L, ed. *Labor und Diagnose*, 8th ed. Bd 1:TH-Books Verlagsgesellschaft 2012.
9. Stein W. *Laboratory Diagnosis of Acute Myocardial Infarction*. Darmstadt: GIT Verlag 1988;34-37.
10. Guder WG, Narayanan S, Wisser H, et al. List of Analytes; Preanalytical Variables. Brochure in: *Samples: From the Patient to the Laboratory*. Darmstadt: GIT-Verlag 1996.
11. Tietz Textbook of Clinical Chemistry and Molecular Diagnostics. Burtis, C.A., Ashwood, E.R., Bruns, D.E.; 5th edition, WB Saunders Comp., 2012.
12. Stein W. Creatine kinase (total activity), creatine kinase isoenzymes and variants. In: Thomas L, ed. *Clinical laboratory diagnostics*. Frankfurt: TH-Books Verlagsgesellschaft;1998.p.71-80
13. Bablok W, Passing H, Bender R, et al. A general regression procedure for method transformation. Application of linear regression procedures for method comparison studies in clinical chemistry, Part III. *J Clin Chem Clin Biochem* 1988 Nov;26(11):783-790.
14. Sonntag O, Scholer A. Drug interference in clinical chemistry: recommendation of drugs and their concentrations to be used in drug interference studies. *Ann Clin Biochem* 2001;38:376-385

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