

Installation of Consumable Management User-Tool Setup

Pre-requisites:

MultiXL (XL Analyzer software) application should be installed on the Target PC.
User should have **administrator** privileges.

Procedure to Install:

Open folder “**Setup**” in the Software Installation Disk or Folder. Perform following steps.

Step 1: Double-Click on “**setup.exe**” to start installation on the Analyzer PC.

Step 2: Click **Next** button. Screen to Select Installation Folder will appear.

Step 3: Default Installation folder is **C:\Program Files\MultiXL**.

Change the folder **only if** MultiXL Installation folder is different from the Default Folder.

To change the folder, click Browse... button and Select the folder at which MultiXL is installed.

Click **Next** button to continue with the installation.

Step 4: **Confirm Installation** screen will appear to start the installation.

Click **Next** button to continue installation.

Step 5: **Installation Complete** screen appears, indicating that software is installed successfully.

Click **Close** button on the screen.

ConsumableManagement option is added to Windows Start menu.

Shortcut for the software is created on Desktop with icon  .