## Erba Clean I

Cat. No.	Product Name	Packaging (Content)
EHL00042	Erba Clean I	6 × 50 mL, instruction for use

### INTENDED USE

Erba Clean I is the washing solution for hydraulic system of automatic coagulation analyser ECL 760 and EC 90 instrument intended for *in vitro* diagnostics. For professional use in clinical laboratory only.

### PRINCIPLE

Erba Clean I is an alkaline solution that effectively cleans and stabilizes flow parts of the analyser. Provides clean environment and defends contamination of samples.

### DESCRIPTION AND COMPOSITION

Active Ingredients: Sodium Hypochlorite 1 %

### PREPARATION

Ready to use. Avoid contamination.

### MATERIAL REQUIRED BUT IS NOT PROVIDED WITH THE DEVICE

Coagulation Analyser ECL 760, Cat. No. INS00070

EC 90 analyser, Cat. No. INS00079

### STABILITY AND STORAGE

Store the solution in a dark place at 2–8 °C (do not freeze).

Before use, keep the solution at room temperature for at least 2 hours.

The solution is stable till the expiry date mentioned on the label.

The solution is stable for 1 month after opening the bottle.

Keep the bottle tightly closed and store the solution in the original bottle.

### ASSAY PROCEDURE

The Erba Clean I should be used in accordance with the instructions in user manual of full automated analyser ECL 760.

### WARNING AND PRECAUTIONS

For *in vitro* diagnostic use. To be handled by entitled and professionally educated person.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

### Hazards identification in accordance with Regulation (EC) No 1272/2008

UFI: TU4W-5JGP-1E72-0XK4

#### Warning

Contains: Sodium hypochlorite

#### Hazard statement:

H315 Causes skin irritation.

H319 Causes serious eye irritation.

H412 Harmful to aquatic life with long lasting effects.

#### Precautionary statement:

P273 Avoid release to the environment.

P280 Wear protective gloves / protective clothing / eye protection.

P302+P352 IF ON SKIN: Wash with plenty of water.

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

### WASTE MANAGEMENT

Please refer to local legal requirements.

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