Product name	REF	Σ	Specific gravity	Leucocytes	Nitrite	pH	Ascorbic acid	Protein	Glucose	Ketones	Urobilinogen	Bilirubin	Blood
ALBUPHAN [®]	URPH0001	50				■		■					
GLUKOPHAN [®]	URPH0002	50							■				
HEMOPHAN [®]	URPH0003	50											■
KETOPHAN [®]	URPH0004	50								■			
DIAPHAN [®]	URPH0005	50							■	■			
TRIPHAN [®]	URPH0007	50				■		■	■				
PENTAPHAN [®]	URPH0010	50				■		■	■	■			■
HEXAPHAN [®]	URPH0011	50				■		■	■	■			■
HEXAPHAN [®]	URPH0012	100				■		■	■	■			■
HEPTAPHAN [®]	URPH0013	50				■		■	■	■		■	■
HEPTAPHAN [®]	URPH0014	100				■		■	■	■		■	■
NONAPHAN [®] SG	URPH0015	50	■			■		■	■	■			■
NEFROPHAN [®] LEUCO	URPH0016	50		■	■	■		■	■	■			■
DEKAPHAN [®] LEUCO	URPH0017	50	■	■	■	■		■	■	■			■
DEKAPHAN [®] LEUCO	URPH0018	100	■	■	■	■		■	■	■			■
UNDEKAPHAN [®]	URPH0019	50	■	■	■	■	■	■	■	■			■

INTENDED USE

Diagnostic strips for *in vitro* visual semiquantitative determination of specific gravity, leucocytes, nitrites, pH, ascorbic acid, protein, glucose, ketones, urobilinogen, bilirubin and blood in urine. Intended for screening and monitoring of kidney function disturbances, leukocyturia, bacteriuria, acidosis / alkalosis, presence of ascorbic acid, proteinuria, glucosuria, ketonuria, urobilinogenuria, bilirubinuria and haematouria. Ascorbic acid in urine may affect the determination of other parameters (blood, glucose, nitrite). For professional use at clinical laboratories only. The semiquantitative analysis is not sufficient for the completing of diagnoses.

DESCRIPTION

For visual evaluation there are available 11 indication zones (specific gravity, leucocytes, nitrites, pH, protein, glucose, ketones, urobilinogen, bilirubin, blood and ascorbic acid) that are completed either individually as monofunctional strips or in various combinations as polyfunctional strips – all types of available products are mentioned in the table above.

The diagnostic strips are intended for single use only.

SPECIMEN COLLECTION AND HANDLING

For specimen collection and preparation use only suitable clean and dry tubes or containers without traces of detergents and disinfectants.

Do not add the preservatives to the urine.

Use freshly voided urine, well mixed that has not been centrifuged. The urine should not be more than 2 hours old when tested. In case of longer standing, mix before use.

Do not expose the sample into the direct sunlight as this may cause the degradation of bilirubin and urobilinogen and this may turn in lower or false negative results.

Mind that woman menstruation blood may affect the results.

ASSAY PROCEDURE

- Remove only as many test strip required and reseal the tube immediately after use. Do not touch test pads of the strip. Do not divide test strips!
- Immerse completely all reagent pads in urine specimen (for 1–2 seconds).
- Run edge of the strip against rim of urine container to remove excess urine and wipe off the excess urine from the strip to the cellulose cotton wool as well. Hold strip in horizontal position.
- After. ca. 60 seconds (120 seconds in case of leucocytes) compare the test pads to corresponding colour scale on the label.

For increasing the accuracy of colour reading when the resulting colour lies between the two neighbouring colour specifications on the scale, there is possible to approximate the concentration according to the intensity of the test zone, or to count the concentration as the arithmetic average from the concentrations assigned to these neighbouring colours on the scale.

EVALUATION

After immersion, while in contact with individual urine components, diagnostic zones change their colour which is compared visually with the comparative colour scale on the tube label. The intensity and shade of zone coloration are proportional to the concentration of analyte.

WORKING REAGENT CONCENTRATIONS

Specific gravity: poly(methylvinyl ether/maleic acid) 32 %; bromthymol blue 5.1 %

Leucocytes: indoxyl ester 0.43 %; diazonium salt 0.05 %

Nitrite: sulphanylamide 5.1 %; tetrahydrobenzo-[h]quinoline 5.8 %

pH: methyl red 0.71 %; bromthymol blue 12.1 %

Ascorbic Acid: phosphomolybdic acid 26 %

Protein: tetrabromphenolphthalein ester 0.21 %; tetrabromphenol blue 0.35 %

Glucose: glucose oxidase 1.3 %; ascorbate oxidase 0.7 %; peroxidase 1.3 %; tetramethylbenzidine 21 %

Ketones: sodium nitroprusside 4.9 %

Urobilinogen: diazonium salt 2.3 %

Bilirubin: diazonium salt 0.75 %

Blood: tetramethylbenzidine 1.5 %; cumene hydroperoxide 15.2 %

PRINCIPLE

Specific gravity – The test is based on the principle of ion exchange, which runs between polyelectrolyte and ions present in urine. Its result is colour change of acid-base indicator from blue-green colour in urine with low concentration of ions, through green and yellow-green in urine with increased concentration of ions, to amber yellow colour.

Leucocytes – The test is based on enzymatic reaction. The test pad contains an indoxyl ester that is cleaved by granulocyte esterases. The released indoxyl reacts with a diazonium salt and pink or violet coloration is formed.

Nitrites – The test is based on conversion of nitrate to nitrite by the action of certain species of bacteria contained in urine. The colour test is based on the principle of the Griess test. Any degree of pink coloration should be interpreted as a positive nitrite test.

pH – The test is based on the double acid-base indicator principle and gives a range of colours from orange through yellow and green to blue.

Ascorbic Acid – The test is based on the reaction of phosphomolybdic acid which is reduced by ascorbic acid to molybdenum blue.

Protein – The test is based on colour change of acid-base indicator, which is caused by presence of proteins. It is particularly sensitive to albumin, but is much less sensitive to globulin, mucoprotein, haemoglobin and Bence-Jones protein.

Glucose – The test is based on the specific glucose oxidase/peroxidase reaction. The test is specific only for D-glucose.

Ketones – The test is based on the principle of Legal's test and is more susceptible to acetoacetic acid than to acetone. Test does not react with β-hydroxybutyric acid.

Urobilinogen – The test is based on the coupling of urobilinogen with stabilized reagent. The test is specific for urobilinogen and stercobilinogen and is not susceptible to the interfering factors known at Ehrlich's test.

Bilirubin – The test is based on the coupling of bilirubin with stabilized reagent. Even slightly orange coloration of the diagnostic pad means pathologic result.

Blood – The test is based on the peroxidase activity of haemoglobin which catalyzes the oxidation of the indicator due to the presence of the organic hydroperoxide contained in the diagnostic pad. The label contains two colour scales; for detection of intact erythrocytes and free haemoglobin. The test is highly sensitive to free haemoglobin.

LIMITATIONS

Specific gravity – This reaction is affected by pH values of urine over 6.5, this can cause shift in colour response towards lower values of specific gravity.

Leucocytes – In case a urine sample is more markedly coloured (e.g. increased bilirubin content), the resulting colour could be affected by a sample coloration. The intensity of colour reaction is increased by alkaline pH and higher urine density.

Nitrite – Before testing the patient should intake vegetable-rich meals and discontinue antibiotic therapy and vitamin C for 3 days prior to the test. Sensitivity of this test declines with high specific gravity of urine. Increased diuresis can cause false negative results. Limited fluid intake prior testing can prevent from the excessive dilution of urine. Test can be applied only at fresh urine. Inaccurate results may occur at stale urines, in which nitrite can be formed by contamination of the specimen.

Protein – In strongly alkaline urines (pH >8) from patients on medication with quinine or quinoline containing drugs false positive reading may be obtained. False positive results may be found when the urine collection vessel contains traces of disinfectants with quaternary ammonium groups. On the other hand, in the presence of non-ionic or anionic detergents, false-negative results may occur. Do not take the colour of the dry pad into consideration.

Glucose – The reaction is independent on pH and presence of ketone bodies.

Ketones – Drugs and diagnostics on the basis of phenolphthalein or sulphophthalein may turn red to purple because of alkaline reaction of the pad.

Urobilinogen – The reaction is not affected by pH of urine. The presence of bilirubin gives yellow colour. This colour which turns slowly to greenish-blue does not interfere with urobilinogen determination provided the reading is made 1 minute after wetting. The urine specimen should not be exposed to direct sunlight as this promotes the oxidation of urobilinogen and thus leads to artificially low or false negative readings.

Bilirubin – The urine specimen should not be exposed to direct sunlight as this promotes the oxidation of bilirubin and thus leads to artificially low or false negative readings. High concentrations of urobilinogen (above 100 µmol/L) interfere with the test. Also the red substances or the substances, which are turning red in low pH may interfere (e.g. Phenazopyridine). The reaction is not affected by pH of urine.

Blood – Microbial peroxidase associated with urinary tract infection may cause a false positive reaction. The sensitivity of the test is influenced by specific gravity or by inhibitors of pharmacological origin.

All diagnostics pads do not interfere with the common concentration of the ascorbic acid.

PLEASE NOTE

Knowledge of the effects of drugs or their metabolites upon the individual tests is not yet complete. In doubtful cases, it is advisable to repeat the test after discontinuing a drug. The sensitivity depends upon the variability of urines. The semiquantitative analysis is not sufficient for the completing of diagnoses.

MATERIAL REQUIRED BUT NOT PROVIDED

Control urines URINORM (Cat.No. REG00053), or Bio-Rad Liquichek, stop watches.

QUALITY CONTROL

For quality control of precision and accuracy URINORM control urines, Cat.No. REG00053 are recommended. Obtained results are compared with the Target values in the instruction for use of URINORM. Bio-Rad Liquichek Urinalysis Control can be also used for quality control. It is recommended to perform QC measurements according to your local laboratory guidelines.

STABILITY AND STORAGE

Keep diagnostic test strips in tightly closed original tubes in a dry and dark place at 2–30 °C. The strips must be kept away from moisture, direct sunlight, elevated temperature and chemical fumes in the laboratory. When stored under these conditions, test strips are stable to the expiry date given on the pack. Once opened, the diagnostic strips are stable for at least 3 months.

ANALYTICAL PERFORMANCE

Limit of detection is defined as the lowest concentration of the analyte which leads to positive results in ≥90 % cases. Limit of detection is not applicable for SG and pH.

Method comparison of PHAN diagnostic strips was performed against COBAS U411 analyzer and Combur 10 test M. All concentration ranges were covered.

Parameter	Result Values			Limit of detection	Method Comparison
SG	1.000, 1.005, 1.010, 1.015, 1.020, 1.025, 1.030			NA	88 %*
LEU	neg., 10–25, 75, 500 LEU/µL	neg., 1+, 2+, 3+		25 LEU/µL	neg. 90 % pos. 98 %
NIT	neg., pos.			0.8 mg/L	neg. 100 % pos. 100 %
pH	5, 6, 7, 8, 9			NA	99 %*
ASC	10, 20, 40, 60 mg/dL	0.6, 1.1, 2.3, 3.4 mmol/L		4 mg/dL	NA
PRO	neg., 30, 100, 500 mg/dL	neg., 0.30, 1, 5 g/L	neg., 1+, 2+, 3+	0.22 g/L	neg. 98 % pos. 100 %
GLU	neg., 50, 100, 300, 1000 mg/dL	neg., 2.8, 5.5, 17, 55 mmol/L	neg., 1+, 2+, 3+, 4+	1.7 mmol/L	neg. 100 % pos. 100 %
KET	neg., 16, 52, 156 mg/dL	neg., 1.5, 5, 15 mmol/L	neg., 1+, 2+, 3+	0.3 mmol/L	neg. 99 % pos. 95 %
UBG	norm., 1, 3, 6, 12 mg/dL	norm., 17, 51, 102, 203 µmol/L	norm., 1+, 2+, 3+, 4+	0.8 mg/dL	neg. 100 % pos. 88 %
BIL	norm., 1+, 2+, 3+			0.8 mg/dL	neg. 100 % pos. 83 %
BLD	neg., 5–10, 50, 250 ERY/µL	neg., 1+, 2+, 3+		7 ERY/µL	neg. 99 % pos. 90 %

* The acceptance criteria for SG and pH are that the satisfactory results may lay in ±1 concentration field.

Precision

Precision testing consists of Intra-assay and Inter-assay measurements using control urines material.

Intra-assay - Two levels of control urine material was used as a sample. Each level was measured 20 times in one day.

Inter-assay - Two levels of control urine material was used as a sample. Each level was measured 2 times in one day and this testing was repeated in 20 consecutive days.

For both assays control urines BIORAD Liquichek was used as a control.

PRECISION					
Parameter	Control	Intra-assay		Inter-assay	
		Result	Agreement	Result	Agreement
SG	Level 1	1.015	100 %	1.015	100 %
	Level 2	1.010	100 %	1.010	100 %
LEU	Level 1	neg.	100 %	neg.	100 %
	Level 2	500	100 %	500	100 %
NIT	Level 1	neg.	100 %	neg.	100 %
	Level 2	pos.	100 %	pos.	100 %
pH	Level 1	6	100 %	6	100 %
	Level 2	7	100 %	7	100 %
PRO	Level 1	neg.	100 %	neg.	100 %
	Level 2	100	100 %	100	100 %
GLU	Level 1	neg.	100 %	neg.	100 %
	Level 2	300	100 %	300	100 %
KET	Level 1	neg.	100 %	neg.	100 %
	Level 2	16	100 %	16	100 %
UBG	Level 1	neg.	100 %	neg.	100 %
	Level 2	6	100 %	6	100 %
BIL	Level 1	neg.	100 %	neg.	100 %
	Level 2	+++	100 %	+++	100 %
BLD	Level 1	neg.	100 %	neg.	100 %
	Level 2	250	100 %	250	100 %

EXPECTED VALUES

Based on literature.^[1]

Blood:	up to 5 Ery/µL	Glucose:	up to 1.4 mmol/L
Leucocytes:	up to 10 Leu/µL	Nitrite:	10 ³ bacteria/mL
pH:	5.5– 7.0	Bilirubin:	up to 17 µmol/L
Urobilinogen:	up to 1 mg/dL	Protein:	over 30 mg/24 hours
Ketones:	up to 0.19 mmol/L	Specific gravity:	1.015– 1.025

Each laboratory should verify that it is appropriate to apply these values to its patient population and if necessary determine its own ranges.

WARNING AND PRECAUTIONS

For *in vitro* diagnostic use. To be handled by entitled and professionally educated person.

Do not touch test pads of the strip.

The strips must be kept away from moisture, direct sunlight, elevated temperature and chemical fumes in the laboratory.

Do not use in case of color vision disorders, otherwise the results may be affected.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

Hazards identification in accordance with Regulation (EC) No 1272/2008

The product is not classified as dangerous.

WASTE MANAGEMENT

Used strip should be treated as potentially infectious and should be liquidated in accordance with local and national regulations relating to the safe handling of such materials. Let waste recycle or put it to municipal waste.

REFERENCES

- Tietz Textbook of Clinical Chemistry and Molecular Diagnostics. Burtis, C.A., Ashwood, E.R., Bruns, D.E.; 5th edition, WB Saunders Comp., 2012.


USED SYMBOLS

REF Catalogue number

Manufacturer

LOT Lot number

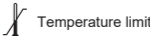Temperature limit

Do not re-use

Expiry date

eFU:
www.erba.com

In vitro diagnostic medical device

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