# CALCIUM

Cat. No.	Pack Name	Packaging (Content)
XSYS0007	CA 120	R1: 10 × 12 mL, RFID tag, instruction for use

## INTENDED USE

The kit is intended for *in vitro* photometric quantitative determination of calcium in human serum, plasma and urine on automatic systems ERBA XL. In combination with other parameters it is intended for screening, monitoring of neuromuscular function, changes of bone metabolism, treatment monitoring. For professional use in clinical laboratories only.

## CLINICAL SIGNIFICANCE

Calcium is the most abundant mineral element in the body with about 99 percent in the bones primarily as hydroxyapatite. The remaining calcium is distributed between the various tissues and the extracellular fluids where it performs a vital role for many life sustaining processes. Among the extra skeletal functions of calcium are involvement in blood coagulation, neuromuscular conduction, excitability of skeletal and cardiac muscle, enzyme activation, and the preservation of cell membrane integrity and permeability.

Serum calcium levels and hence the body content are controlled by parathyroid hormone (PTH), calcitonin, and vitamin D. An imbalance in any of these modulators leads to alterations of the body and serum calcium levels.

Hypercalcaemia may develop in patients with Paget's disease of bone and hyperparathyroidism. The cause of hypercalcaemia in malignancy is an increased bone resorption either caused by metastasis or by humoral factors produced by the tumor cell.

In Rickets, Coeliac diseases, idiopathic steatorrhea, osteomalacia, tropical sprue and following surgical resection of the small intestine, serum calcium is often moderately reduced, usually in association with low plasma protein concentration.

## PRINCIPLE

Calcium with Arsenazo III at neutral pH yields a blue colored complex, whose intensity is proportional to the calcium concentration<sup>1,2</sup>. Interference by magnesium is eliminated by addition of 8-hydroxyquinoline-5-sulfonic acid.

The resulting absorbance at 660 nm is proportional to the calcium concentration in the sample.

## REAGENT DESCRIPTION AND COMPOSITION

R1	Concentration
Phosphate buffer (pH 7.8)	50.0 mmol/L
8-hydroxyquinoline-5-sulfonic acid	5.00 mmol/L
Arsenazo III	0.10 mmol/L

## COMPOSITION OF REACTION MIXTURE

Phosphate buffer (pH 7.8)	49.5 mmol/L
8-hydroxyquinoline-5-sulfonic acid	4.95 mmol/L
Arsenazo III	0.099 mmol/L

## REAGENT PREPARATION

Reagents are liquid, ready to use. Load the number of tests from the RFID tag before using a new kit.

## MATERIAL REQUIRED BUT NOT PROVIDED WITH THE DEVICE

XL MULTICAL 4x3, Cat. No. XSYS0034  
 XL MULTICAL 10x3, Cat. No. XSYS0122  
 ERBA NORM 4x5, Cat. No. BLT00080  
 ERBA NORM 10x5, Cat. No. XSYS0123  
 ERBA PATH 4x5, Cat. No. BLT00081  
 ERBA PATH 10x5, Cat. No. XSYS0124  
 Erba XL analysers: XL-200, Cat. No. INS00002  
 XL-640, Cat. No. INS00008  
 XL-1000, Cat. No. INS00010

## STABILITY AND STORAGE

The unopened reagents are stable till the expiry date stated on the bottle and kit label when stored at 2–8 °C.

On board stability: min. 60 days if refrigerated (2–10 °C) and not contaminated.

## SPECIMEN COLLECTION AND HANDLING

It is recommended to follow ISO 15189 and laboratory instruction. For specimen collection and preparation only use suitable tubes or collection containers. Only the specimens listed below were tested and found acceptable.

Serum: Fresh serum collected in the fasting state is the preferred specimen.

Plasma: Li-heparin

Serum or plasma should be separated from blood cells as soon as possible, because prolonged contact with the clot may cause lower calcium values<sup>6</sup>. Sera from patients receiving EDTA (treatment of hypercalcaemia) are unsuitable for analysis, since EDTA will chelate the calcium and render it unavailable for reaction with Arsenazo III. Co-precipitation of calcium with fibrin (i.e. heparin plasma), lipids, or denatured protein has been reported with storage or freezing<sup>3,7</sup>.

Urine: Urine specimens should be collected in acid-washed bottles. 24-hour specimens should be collected in containers containing 20–30 mL of 6 mol/L HCl to prevent calcium salt precipitation. Precipitated calcium salts may not be completely dissolved by the addition of HCl following urine collection<sup>8</sup>. Dilute urine samples using distilled water in 1+2 ration and multiply results by 3.

The sample types listed were tested with a selection of sample collection tubes that were commercially available at the time of testing, i.e. not all available tubes of all manufacturers were tested. Sample collection systems from various manufacturers may contain differing materials which could affect the test results in some cases. When processing samples in primary tubes (sample collection systems), follow the instructions of the tube manufacturer.

Centrifuge samples containing precipitates before performing the assay. See the Limitations and Interferences section for details about possible sample interferences.

Stability in serum / plasma <sup>9</sup> :	Temperature	Duration
	15–25 °C	7 days
	2–8 °C	3 weeks
	-20 °C	8 months

Stability in urine <sup>9</sup> :	Temperature	Duration
	15–25 °C	2 days
	2–8 °C	4 days
	-20 °C	3 weeks

Discard contaminated specimens.

## CALIBRATION

Calibration with calibrator XL MULTICAL is recommended. 2 point calibration (blank and calibrator); distilled water is recommended as blank. Calibration frequency: 30 days

Calibration is needed:

- after reagent lot change
- as required by internal quality control procedures
- calibration interval may be extended based on acceptable verification of calibration by the laboratory

## QUALITY CONTROL

For quality control ERBA NORM and ERBA PATH are recommended. The control intervals and limits should be adapted according to each individual laboratory's requirements. Values obtained should fall within the defined intervals. Each laboratory should establish corrective measures to be taken if values fall outside the defined limits.

## TRACEABILITY

This method, calibrator XL MULTICAL and controls ERBA NORM and ERBA PATH have been standardized against the SRM 956 reference material.

## ASSAY PROCEDURE AND CALCULATION

ERBA XL automatic systems calculate the concentration of each sample. For assay parameters see [www.erba.com](http://www.erba.com).

## Assay parameters for ERBA XL automatic systems

Assay type	1-Point
Curve type	Linear
Wavelength (prim. / sec.)	660/700 nm
Reading time	90 s after adding of R1
Reaction direction	Increase
Unit	mg/dL (mmol/L)
Reagent volumes	
R1	200 µL
Sample	2 µL

Note: reagents and sample volumes can be different for individual ERBA XL automatic systems depending on the minimum measured volume in the cuvette. The ratio R1:sample does not change.

## UNIT CONVERSION

mg/dL × 0.25 = mmol/L

## EXPECTED VALUES<sup>10</sup>

Serum:	Urine:
Adult: 8.6–10.2 mg/dL	Female: <250 mg/24 h
Children: 0–10 days 7.6–10.4 mg/dL	Male: <300 mg/24 h
10 days–2 years 9.0–11.0 mg/dL	
2–12 years 8.8–10.8 mg/dL	

It is recommended that each laboratory verifies this range or derives reference interval for the population it serves.

## ANALYTICAL PERFORMANCE

Data contained within this section is representative for performance on ERBA XL-640 automatic system. Data obtained in your laboratory may differ from these values. Data for other ERBA XL automatic systems are available on [www.erba.com](http://www.erba.com).

## Limit of quantification:

Serum / plasma	0.45 mg/dL
Urine	4.43 mg/dL

Limit of quantification represents the lowest measurable analyte level. It is calculated as the determined activity of diluted sample to have CV <20 % (n = 30).

## Linearity:

Serum / plasma	20 mg/dL
Urine	60 mg/dL

Linearity is the highest measured activity with recovery within ±10 % from theoretical value.

## Precision:

Precision was determined by using controls in an internal protocol with repeatability (n = 20) and intermediate precision (2 aliquots per run, 2 run per day, 20 days). The following results were obtained:

Repeatability (serum)	Mean (mg/dL)	SD (mg/dL)	CV (%)	Intermediate precision (serum)	Mean (mg/dL)	SD (mg/dL)	CV (%)
Sample 1	8.22	0.130	1.58	Sample 1	8.28	0.206	2.49
Sample 2	10.41	0.076	0.73	Sample 2	10.76	0.164	1.53

Repeatability (urine)	Mean (mg/dL)	SD (mg/dL)	CV (%)	Intermediate precision (urine)	Mean (mg/dL)	SD (mg/dL)	CV (%)
Sample 1	13.8	0.52	3.76	Sample 1	23.1	1.02	4.43
Sample 2	28.9	0.90	3.11	Sample 2	32.5	1.36	4.19

## Accuracy

Two different validated control materials for serum and urine were used. Determined bias is 2.0 % at the target value 11.7 mg/dL, 2.0 % at the target value 12.1 mg/dL for serum, 5.5 % at the target value 8.2 mg/dL and 8.6 % at the target value 12.2 mg/dL for urine.

## Comparison

A comparison between XL-640 automatic system CALCIUM (y) and a commercially available test (x) using 90 samples (serum) gave following results:

Linear regression:  
 $y = 0.936x + 0.765 \text{ mg/dL}$   $r = 0.981$

Passing-Bablok<sup>11</sup>:  
 $y = 0.947x + 0.629 \text{ mg/dL}$   $r = 0.973$

## Interferences

Criterion: Recovery within ±10 % of initial value of calcium concentration in the sample (serum) without interfering substance.

Following substances do not interfere: haemoglobin up to 12.5 g/L, bilirubin up to 40 mg/dL, triglycerides up to 850 mg/dL.

Drugs: No interference in serum or urine was found at therapeutic concentrations using common drug panels<sup>12</sup>.

## Limitations:

- Deteriorated reagents (e.g. exceeding the storage temperature) may give incorrect results. Quality of reagents is monitored on automatic systems ERBA XL by checking of the maximum permissible absorbance value of blank.
- High concentration of haemoglobin, bilirubin and triglycerides in sample can interfere with determination of calcium. See paragraph Interferences.

## WARNING AND PRECAUTIONS

For *in vitro* diagnostic use. To be handled by entitled and professionally educated person. Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

## Hazards identification in accordance with Regulation (EC) No 1272/2008

R1  
 Reagent is not classified as dangerous.

## WASTE MANAGEMENT

Please refer to local legal requirements.

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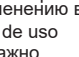
1. Bauer P.J. Affinity and stoichiometry of calcium binding by arsenazo III. *Anal Biochem*, 110: 61-72, 1981.
2. Michaylova V, Ilkova P. Photometric determination of micro amounts of calcium with arsenazo III. *Anal Chim Acta*, 53:194-198, 1971.
3. Endres DB, Rude RK. Mineral and Bone Metabolism. In: Burtis CA, Ashwood ER, Bruns ED, eds. *Tietz Textbook of Clinical Chemistry and Molecular Diagnostics*, 4th ed. St. Louis (MO): Saunders Elsevier: 1891-1965, 2006.
4. Beeler M. F. and Catrou P.G. "Disorders of Calcium Metabolism" in *Interpretations in Clinical Chemistry A.C.S.P. Press Chicago*, 34-44, 1983.
5. Farrell C.E. "Electrolytes" in *Clinical Chemistry Theory. Analysis and Correlation. The C.V. Mosby Company. Kaplan L.A., Pesce A.J. (Ed.)*, Chap 55, 1054, 1984
6. Heins M, Heil W, Withold W. Storage of Serum or Whole Blood Samples? Effect of Time and Temperature on 22 Serum Analytes. *Eur J Clin Chem Clin Biochem*, 33:231-238, 1995.
7. Wilding P, Zilva JF, Wilde CE. Transport of specimens for clinical chemistry analysis. *Ann Clin Biochem*, 14:301-306, 1977.
8. Burtis CA, Ashwood ER, Bruns DE, eds. *Tietz Fundamentals of Clinical Chemistry*, 6th ed. St. Louis (MO): Saunders Elsevier, 715, 2008.
9. Use of Anticoagulants in Diagnostic Laboratory Investigations. WHO Publication WHO/DIL/LAB/99.1 Rev. 2: Jan 2002.
10. *Tietz Textbook of Clinical Chemistry and Molecular Diagnostics*. Burtis, C.A., Ashwood, E. R., Bruns, D.E.; 5th edition, WB Saunders Company, 2012.
11. Bablok W, Passing H, Bender R, et al. A general regression procedure for method transformation. Application of linear regression procedures for method comparison studies in clinical chemistry, Part III. *J Clin Chem Clin Biochem* 1988 Nov; 26(11): 783-790.
12. Sonntag O, Scholer A. Drug interference in clinical chemistry: recommendation of drugs and their concentrations to be used in drug interference studies. *Ann Clin Biochem* 2001;38:376-385

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## LITERATÚRA

1. Bauer P.J. Affinity and stoichiometry of calcium binding by arsenazo III. *Anal Biochem*, 110: 61-72, 1981.
2. Michaylova V, Ilkova P. Photometric determination of micro amounts of calcium with arsenazo III. *Anal Chim Acta*, 53:194-198, 1971.
3. Endres DB, Rude RK. Mineral and Bone Metabolism. In: Burtis CA, Ashwood ER, Bruns ED, eds. *Tietz Textbook of Clinical Chemistry and Molecular Diagnostics*, 4th ed. St. Louis (MO): Saunders Elsevier: 1891-1965, 2006.
4. Beeler M. F. and Catrou P.G. "Disorders of Calcium Metabolism" in *Interpretations in Clinical Chemistry* A.C.S.P. Press Chicago, 34-44, 1983.
5. Farrell C.E. "Electrolytes" in *Clinical Chemistry Theory. Analysis and Correlation*. The C.V. Mosby Company. Kaplan L.A., Pesce A.J. (Ed.), Chap 55, 1054, 1984
6. Heins M, Heil W, Withold W. Storage of Serum or Whole Blood Samples? Effect of Time and Temperature on 22 Serum Analytes. *Eur J Clin Chem Clin Biochem*, 33:231-238, 1995.
7. Wilding P, Zilva JF, Wilde CE. Transport of specimens for clinical chemistry analysis. *Ann Clin Biochem*, 14:301-306, 1977.
8. Burtis CA, Ashwood ER, Bruns DE, eds. *Tietz Fundamentals of Clinical Chemistry*, 6th ed. St. Louis (MO): Saunders Elsevier, 715, 2008.
9. Use of Anticoagulants in Diagnostic Laboratory Investigations. WHO Publication WHO/DIL/LAB/99.1 Rev. 2: Jan 2002.
10. *Tietz Textbook of Clinical Chemistry and Molecular diagnostics*. Burtis, C.A., Ashwood, E. R., Bruns, D.E.; 5th edition, WB Saunders Company, 2012.
11. Bablok W, Passing H, Bender R, et al. A general regression procedure for method transformation. Application of linear regression procedures for method comparison studies in clinical chemistry, Part III. *J Clin Chem Clin Biochem* 1988 Nov; 26(11): 783-790.
12. Sonntag O, Scholer A. Drug interference in clinical chemistry: recommendation of drugs and their concentrations to be used in drug interference studies. *Ann Clin Biochem* 2001;38:376-385

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