BILIRUBIN TOTAL

Cat. No.	Pack Name	Packaging (Content)
XSYS0023	BIL T 330	R1: 6 × 44 mL, R2: 6 × 11 mL, RFID tag, instruction for use

INTENDED USE

The kit is intended for *in vitro* photometric quantitative determination of bilirubin total in human serum and plasma on automatic systems ERBA XL. In combination with other parameters it is intended for screening, monitoring and diagnosis of liver diseases. For professional use in clinical laboratory only.

CLINICAL SIGNIFICANCE

Red blood cells at the end of their circulating lives are broken down in the reticuloendothelial system, mainly the spleen. The resulting heme is converted to bilirubin upon removal of iron. This process accounts for about 80% of the 500 µmol (292 mg) of bilirubin formed daily. Other sources of bilirubin include the breakdown of myoglobin and cytochromes and the catabolism of immature red blood cells in the bone marrow.

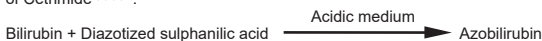
Once formed, bilirubin is transported to the liver bound to albumin. This fraction of bilirubin is referred to as indirect or unconjugated bilirubin. In the liver, bilirubin is conjugated to glucuronic acid (mono- and diglucuronides) by the enzyme uridyl diphosphate glucuronyl transferase to form conjugated bilirubin. Conjugated bilirubin or direct bilirubin is excreted via the biliary system into the intestine, where it is metabolized by bacteria to a group of products known collectively as sterco-bilinogen. Elimination is almost complete and serum levels are normally negligible.

Total bilirubin is the sum of the unconjugated and conjugated fractions. Total bilirubin is elevated in hepatitis, cirrhosis, hemolytic disorders, several inherited enzyme deficiencies, and conditions causing hepatic obstruction.

TOTAL BILIRUBIN = INDIRECT BILIRUBIN + DIRECT BILIRUBIN

PRINCIPLE

Bilirubin and bilirubin glucuronate react with diazotized sulphanilic acid in a strongly acidic medium and form an intensely coloured diazo dye – azobilirubin. Bilirubin glucuronate is soluble in water and reacts directly, bilirubin associated with albumin must be previously hydrolysed in presence of Cetrimide^{1,2,3,4,5}.

Absorbance of the Azobilirubin measured at 546 nm is proportional to the concentration of the total bilirubin in the sample.

REAGENT DESCRIPTION AND COMPOSITION

R1	R2
Sulphanilic acid 28.9 mmol/L	Sodium nitrite 2.90 mmol/L
HCl 58.8 mmol/L	
Cetrimide 68.6 mmol/L	

COMPOSITION OF REACTION MIXTURE

Sulphanilic acid	22.0 mmol/L
HCl	44.8 mmol/L
Cetrimide	52.2 mmol/L
Sodium nitrite	0.55 mmol/L

REAGENT PREPARATION

Reagents are liquid, ready to use. Load the number of tests from the RFID tag before using a new kit.

MATERIAL REQUIRED BUT NOT PROVIDED WITH THE DEVICE

XL MULTICAL 4×3, Cat. No. XSYS0034
 XL MULTICAL 10×3, Cat. No. XSYS0122
 ERBA NORM 4×5, Cat. No. BLT00080
 ERBA NORM 10×5, Cat. No. XSYS0123
 ERBA PATH 4×5, Cat. No. BLT00081
 ERBA PATH 10×5, Cat. No. XSYS0124
 Erba XL analysers: XL-200, Cat. No. INS00002
 XL-640, Cat. No. INS00008
 XL-1000, Cat. No. INS00010

STABILITY AND STORAGE

The unopened reagents are stable till the expiry date stated on the bottle and kit label when stored at 2–8 °C.
 On board stability: min. 60 days if refrigerated (2–10 °C) and not contaminated.

SPECIMEN COLLECTION AND HANDLING

It is recommended to follow ISO 15189 and laboratory instruction.
 For specimen collection and preparation only use suitable tubes or collection containers.
 Only the specimens listed below were tested and found acceptable.
 Serum.

Plasma: Li-heparin and K₂-EDTA plasma.

The sample types listed were tested with a selection of sample collection tubes that were commercially available at the time of testing, i.e. not all available tubes of all manufacturers were tested. Sample collection systems from various manufacturers may contain differing materials which could affect the test results in some cases. When processing samples in primary tubes (sample collection systems), follow the instructions of the tube manufacturer.

Centrifuge samples containing precipitates before performing the assay.

See the limitations and interferences section for details about possible sample interferences.

Stability in serum / plasma:

1 day at	15–25 °C
7 days at	4–8 °C
6 months at	-20 °C

Protect sample from light.
 Discard contaminated specimens.

CALIBRATION

Calibration with calibrator XL MULTICAL is recommended.
 2 point calibration (blank and calibrator); distilled water is recommended as blank
 Calibration frequency: 30 days
 Calibration is needed:
 • after reagent lot change
 • as required by internal quality control procedures
 • calibration interval may be extended based on acceptable verification of calibration by the laboratory

QUALITY CONTROL

For quality control ERBA NORM and ERBA PATH are recommended.
 The control intervals and limits should be adapted according to each individual laboratory's requirements. Values obtained should fall within the defined intervals. Each laboratory should establish corrective measures to be taken if values fall outside the defined limits.

TRACEABILITY

This method, calibrator XL MULTICAL and controls ERBA NORM and ERBA PATH have been standardized against the reference material NIST SRM 916.

ASSAY PROCEDURE AND CALCULATION

ERBA XL automatic systems calculate the concentration of each sample. For assay parameters see www.erba.com.

Assay parameters for ERBA XL automatic systems

Assay type	2-Point
Curve type	Linear
Wavelength (nm/sec.)	546/660 nm
Reading time 1	just before adding of R2
Reading time 2	10 min after adding of R1
Reaction direction	Increase
Unit	mg/dL (µmol/L)
Reagent volumes	
R1	200 µL
R2	50 µL
Sample volumes	12.6 µL

Note: reagents and sample volumes can be different for individual ERBA XL automatic systems depending on the minimum measured volume in the cuvette. The ratio R1 : R2 : sample does not change.

UNIT CONVERSION

mg/dL × 17.1 = µmol/L

EXPECTED VALUES¹⁰

In serum:	Premature	Full term
Cord	<2.0	<2.0 mg/dL
0–1 d	1.0–8.0	2.0–6.0 mg/dL
1–2 d	6.0–12.0	6.0–10.0 mg/dL
3–5 d	10.0–14.0	4.0–8.0 mg/dL
Adult		0–2.0 mg/dL

It is recommended that each laboratory verifies this range or derives reference interval for the population it serves.

ANALYTICAL PERFORMANCE

Data contained within this section is representative for performance on ERBA XL-640 automatic system. Data obtained in your laboratory may differ from these values. Data for other ERBA XL automatic systems are available on www.erba.com.

Limit of quantification: 0.02 mg/dL

Limit of quantification represents the lowest measurable analyte level. It is calculated as the determined activity of diluted sample to have CV <20 % (n = 30).

Linearity: 30 mg/dL

Linearity is the highest measured activity with recovery within ±10 % from theoretical value.

Precision:

Precision was determined by using controls in an internal protocol with repeatability (n = 20) and intermediate precision (2 aliquots per run, 2 run per day, 20 days). The following results were obtained:

Repeatability	Mean (mg/dL)	SD (mg/dL)	CV (%)
Sample 1	1.56	0.007	0.44
Sample 2	5.25	0.028	0.53

Intermediate precision	Mean (mg/dL)	SD (mg/dL)	CV (%)
Sample 1	1.37	0.027	1.96
Sample 2	4.76	0.094	1.98

Accuracy

Two different validated control materials were used. Determined bias is -0.8 % at the target value 1.68 mg/dL and 6.7 % at the target value 4.55 mg/dL.

Comparison

A comparison between XL-640 automatic system BILIRUBIN TOTAL (y) and a commercially available test (x) using 150 samples gave following results:

Linear regression:
 $y = 1.014x - 0.056 \text{ mg/dL}$ $r = 0.998$

Passing-Bablok¹¹:
 $y = 0.920x + 0.046 \text{ mg/dL}$ $r = 0.916$

Interferences

Criterion: Recovery within ±10 % of initial value of total bilirubin concentration in the sample without interfering substance.

Following substances do not interfere: haemoglobin up to 12.5 g/L, triglycerides up to 850 mg/dL. Drugs: No interference was found at therapeutic concentrations using common drug panels, except positive interferences of Rifampicin¹².

Limitations:

- Deteriorated reagents (e.g. exceeding the storage temperature) may give incorrect results. Quality of reagents is monitored on automatic systems ERBA XL by checking of the maximum permissible absorbance value of blank.
- High concentration of haemoglobin and triglycerides in sample can interfere with determination of total bilirubin. Some drugs can also interfere. See paragraph Interferences.

WARNING AND PRECAUTIONS

For *in vitro* diagnostic use. To be handled by entitled and professionally educated person. Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

Hazards identification in accordance with Regulation (EC) No 1272/2008

R1
 UFI: 0V0F-WE2R-2H7R-9P9K

Danger

Hazard statements:

H314 Causes severe skin burns and eye damage.

Precautionary statements:

- P260 Do not breathe vapours.
- P280 Wear protective gloves/protective clothing/eye protection/face protection.
- P301 + P330 + P331 IF SWALLOWED: Rinse mouth. Do NOT induce vomiting.
- P303 + P361 + P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water or shower.
- P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

Supplemental information

EUH 208 Contains sulphanilic acid. May produce an allergic reaction.

R2

Reagent is not classified as dangerous.

WASTE MANAGEMENT

Please refer to local legal requirements.

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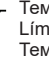
1. Wahlefeld AW, Herz G, Bernt E. Modification of the Malloy-Evelyn method for a simple, reliable determination of total bilirubin in serum. Scand J Clin Lab Invest 29: Supplement 126: Abstract 11.12, 1972.
2. Rand RN, di Pasqua A, A new diazo method for the determination of bilirubin. Clin Chem 6: 570-578, 1962.
3. Mc Phaul L, Kershaw M, Tilque D, J.H. Eckfeldt JH, A 2,4-dichlorophenyl diazonium-based method for total bilirubin without interference from indican in uremic sera, Clin. Chem. 31: 1229-1231, 1985.
4. Dapper GM, Sundberg MW, Wu TW, Babb BE, J.R. Schaeffer JR, A diazo-based dry film for determination of total bilirubin in serum, Clin. Chem. 29: 37-41, 1983.
5. Quality of Diagnostic Samples, Recommendations of the Working Group on Preanalytical Quality of the German Society for Clinical Chemistry and Laboratory medicine, 3rd completely revised ed. 2010.
6. Balistreri WF, Shaw LM. Liver function. In: Tietz NW, ed. Fundamentals of Clinical Chemistry. 3rd ed. Philadelphia: WB Saunders: 729-761, 1987.
7. Thomas L ed. Clinical Laboratory Diagnostics. 1st ed. Frankfurt: TH-Books Verlagsgesellschaft: 192-202, 1998.
8. Tolman KG, Rej R. Liver function. In: Burtis CA, Ashwood ER, editors. Tietz Textbook of Clinical Chemistry. 3rd ed. Philadelphia: W.B Saunders Company; 1125-77, 1999.
9. Dumas BT, Perry BW, Jendrzeczak B, et al. Pitfalls in the American Monitor Kit Methods for Determination of Total and Direct Bilirubin. Clin Chem 28 (11), 2305-2308, 1982.
10. Tietz Textbook of Clinical Chemistry and Molecular Diagnostics. Burtis, C.A., Ashwood, E.R., Bruns, D.E.; 5th edition, WB Saunders Comp., 2012.
11. Bablok W, Passing H, Bender R, et al. A general regression procedure for method transformation. Application of linear regression procedures for method comparison studies in clinical chemistry, Part III. J Clin Chem Clin Biochem, Nov;26(11): 783-790, 1988.
12. Sonntag O, Scholer A. Drug interference in clinical chemistry: recommendation of drugs and their concentrations to be used in drug interference studies. Ann Clin Biochem 2001;38:376-385

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LITERATÚRA

1. Wahlefeld AW, Herz G, Bernt E. Modification of the Malloy-Evelyn method for a simple, reliable determination of total bilirubin in serum. Scand J Clin Lab Invest 29: Supplement 126: Abstract 11.12, 1972.
2. Rand RN, di Pasqua A, A new diazo method for the determination of bilirubin. Clin Chem 6: 570-578, 1962.
3. Mc Phaul L, Kershaw M, Tilque D, J.H. Eckfeldt JH, A 2,4-dichlorophenyl diazonium-based method for total bilirubin without interference from indican in uremic sera, Clin. Chem. 31: 1229-1231, 1985.
4. Dapper GM, Sundberg MW, Wu TW, Babb BE, J.R. Schaeffer JR, A diazo-based dry film for determination of total bilirubin in serum, Clin. Chem. 29: 37-41, 1983.
5. Quality of Diagnostic Samples, Recommendations of the Working Group on Preanalytical Quality of the German Society for Clinical Chemistry and Laboratory medicine, 3rd completely revised ed. 2010.
6. Balistreri WF, Shaw LM. Liver function. In: Tietz NW, ed. Fundamentals of Clinical Chemistry. 3rd ed. Philadelphia: WB Saunders: 729-761, 1987.
7. Thomas L ed. Clinical Laboratory Diagnostics. 1st ed. Frankfurt: TH-Books Verlagsgesellschaft: 192-202, 1998.
8. Tolman KG, Rej R. Liver function. In: Burtis CA, Ashwood ER, editors. Tietz Textbook of Clinical Chemistry. 3rd ed. Philadelphia: W.B Saunders Company; 1125-77, 1999.
9. Dumas BT, Perry BW, Jendrzyczak B, et al. Pitfalls in the American Monitor Kit Methods for Determination of Total and Direct Bilirubin. Clin Chem 28 (11), 2305-2308, 1982.
10. Tietz Textbook of Clinical Chemistry and Molecular Diagnostics. Burtis, C.A., Ashwood, E.R., Bruns, D.E.; 5th edition, WB Saunders Comp., 2012.
11. Bablok W, Passing H, Bender R, et al. A general regression procedure for method transformation. Application of linear regression procedures for method comparison studies in clinical chemistry, Part III. J Clin Chem Clin Biochem, Nov;26(11): 783-790, 1988.
12. Sonntag O, Scholer A. Drug interference in clinical chemistry: recommendation of drugs and their concentrations to be used in drug interference studies. Ann Clin Biochem 2001;38:376-385

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