

## ADVISORY NOTICE

23\_2025\_ Overview H560, H360 PRINTER BOX 50.99.0038A

Dear customer,

We hereby inform you that the H560 (Type A & B) & H360 device can be connected to an external Printer such as:

Options are the following:

1. Direct connection is possible for a limited list of compatible external printers, availability depending on the market. Instruments are Linux based, and the addition of printer drivers is not possible.
  - HP LaserJet Pro M403dw
  - HP LASERJET P1108 (Black and white printer)
  - HP LASERJET P1106 (Black and white printer)
  - HP Deskjet 1010 (Color inkjet printer)
  - HP LASERJET 1020PLUS (Black and white printer)
2. If laboratory owns their own PC and any printer, or our Business Partner provides PC/monitor/printer.
  - Installing "software printer box" on laboratory PC and from there to any printer. This option is free; software can be downloaded from our technical support website. Our Technical support can help with installing and setting up. This option allows customers to use any printer model they wish.
3. The laboratory doesn't own a PC.
  - Installing "hardware Printer box" PN 50006182 which can be purchased from us. Customer or Business Partner needs to provide monitor and printer. This option allows customers to use any printer model they wish

For installation procedure and software download please refer to our [Erba Support platform](#) and navigate to proper files

Applied for:

**INSTRUMENT:**

H360  
H560

cat. no. INS00077  
cat. no. INS00078

Erba Lachema s.r.o.



In case you have any additional questions, please do not hesitate to contact us.

Best regards,

Erba Lachema s.r.o.

A handwritten signature in black ink, appearing to be a stylized name, written over the printed name "Erba Lachema s.r.o." and extending upwards into the main body of the letter.

**Erba Lachema s.r.o.**  
Karásek 2219/1d, 621 00 Brno  
DIČ: CZ26918846 -7-

Brno, 20<sup>th</sup> May 2025

Ref. No.AN\_23\_2025



Erba Lachema s.r.o., Karásek 2219/1d, 621 00 Brno, Czech Republic  
Identification number: 269 18 846. Tax identification number: CZ26918846  
Incorporated in the Commercial Register maintained by the Regional Court in Brno, Section C, insert 45458  
Tel.: +420 517 077 111, e-mail: [diagnostics@erba.com](mailto:diagnostics@erba.com), [www.eralachema.com](http://www.eralachema.com)