

## ADVISORY NOTICE

31\_2025\_XL 180 New software update

Dear customer,

We hereby inform you that the XL180 device software has been updated to SW version 2024.02.

### 1. Purpose:

Application software ver.2024.02 released to include new features and to address issues in previous application software ver.2023.01.

Embedded software ver. XL180\_2024.RE.01 released to include new feature in previous embedded software ver. XL180\_2023.RE.01

### 2. List of changes in Application Software ver. 2024.02:

New feature: Set Rerun with Decrease volume for 'ABS!' Flag (ABS greater than 3.0).

New feature: In Calibration Policy, by Default, Set Discard Paired bottle as Selected for 2 reagent chemistry. [Not applicable for 1 reagent chemistry].

New Feature: Run Report for Traceability.

Bug fixed: After entering the k-factor, Calibration graph is not drawn sometimes.

SW version **2024.02** is already included in devices produced from SN S170018, S170024 onwards.

For SW versions, please contact your local service engineer or distributor.

The latest version of the SW is available on the [ERBA Knowledge Base](#) website.

Applied for:

### INSTRUMENTS:

50006793	INS00037	XL-180 (with ISE)
50006792	INS00038	XL-180 (without ISE)

In case you have any additional questions, please do not hesitate to contact us.

Best regards,  
Erba Lachema s.r.o.

**Erba Lachema s.r.o.**  
Karásek 2219/1d, 621 00 Brno  
DIČ: CZ26918846 -7-

Brno, 4<sup>th</sup> June 2025

Ref. No.AN\_31\_2025

