

ADVISORY NOTICE

33_2025_XL 1000 New software update

Dear customer,

We hereby inform you that the XL 1000 device software has been updated to SW version 2024.06.

1. Purpose:

Application software ver.2024.06 released to include new features, improvement and to address issues reported in previous application software ver.2024.01A. Embedded software ver. ST10R00.09_ESW2024_R0.01 for Old hardware & STM32F4.14_ESW2024_RM.02 for new hardware is released include new feature.

2. List of changes in Application Software ver. 2024.06:

New Feature: For N! Flag, cutoff changed from 0 to -0.01.

Bug Fixed: Auto Copy Calibration is not working for Standards, if scheduled with only one replicate.

New Feature: Stirrer Wash to be performed for COP Wash with Detergent/ Reagent and System Wash.

Bug Fixed: Scanned reagent bottle gets displayed multiple times in Reagents screen for multiple tests using same reagent (e.g. GLU, GLUPP, GLUF etc) including copy test.

SW version **2024.06** is already included in devices produced from SN 115P002 to 115P007.

For SW versions, please contact your local service engineer or distributor.

The latest version of the SW is available on the [ERBA Knowledge Base](#) website.


Applied for:

INSTRUMENTS:

50006796	INS00009	XL-1000 (with ISE)
50006985	INS00010	XL-1000 (without ISE)

In case you have any additional questions, please do not hesitate to contact us.

Best regards,
Erba Lachema s.r.o.

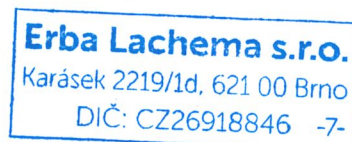

Brno, 5th June 2025

Ref. No.AN_33_2025