

ADVISORY NOTICE

16_2025_Elite 580_ new software version SW A11.4

Dear customers,

We hereby inform you that the ELITE 580 device software has been updated to SW version SW A11.4.

1. Purpose: Bug from previous software solved & New items

2. List of changes in Application Software ver. A11.4

- 1 New research parameters NRBC #, NRBC%, WBC added
- 2 Add Auto QC function
- 3 IG Para. alarm name change
- 4 Low value WBC shielding
- 5 QC printing adding model names to relevant printing reports
- 6 Provide Host and Boot burning programs and tooling
- 7 Auto matching sample in work list
- 8 Optimization of barcode quality for racks
- 9 Fallback of old version card RFMCU version
- 10 NRBC performance optimization
- 11 Add Ukrainian language

SW version A11.4 is already included in devices produced from SN K11052521001.

For SW version A11.4, please contact your local service engineer or distributor.
The latest version of the SW is available on the [ERBA Knowledge Base](#) website.

Applied for:

INSTRUMENTS:

50004675	INS00071	ELite 580
----------	----------	-----------

In case you have any additional questions, please do not hesitate to contact us.

Best regards,
Erba Lachema s.r.o.



Brno, 13th August 2025

Ref. No.AN_16_2025