

ADVISORY NOTICE

44_2025_H7100_ new software version SW A3.0

Dear customers,

We hereby inform you that the H7100 device software has been updated to SW version SW A3.0.

1. Purpose: Bug from previous software solved & New items

2. List of changes in Application Software ver. A3.0:

1) QC interface shielding parameters:

I. CDR mode do 6D-QC, doesn't display RET series parameters (RET #, RET%, LFR, MFR, HFR, IRF, RHE)

II. RET mode do Ret-QC, doesn't display parameters IPF # and IPF%

2) Save calibration history and support viewing detailed calibration information

3) Multilingual support - - Spanish, French, Portuguese

4) Bug fix - Change "Mentzr" to "Mentzer"

5) Worklist function:

I. DMS pre-input function add tube position input, results show tube position

II. CMS mode setting page increase tube position setting

6) Cal results printing function - Analyzer can print all Cal results in "Cal history" page.

7) DMS Sample ID format - Support space and all special characters except Chinese characters and "Λ:*?"<>|".

SW version A3.0 is already included in devices produced from SN K11162523001.

For SW version A3.0, please contact your local service engineer or distributor.

The latest version of the SW is available on the [ERBA Knowledge Base](#) website.

Applied for:

INSTRUMENTS:

50008197	INS00087	H7100 (220-240V) single set
50008485	INS00087	H7100 (115V) single set

In case you have any additional questions, please do not hesitate to contact us.

Best regards,
Erba Lachema s.r.o.

Erba Lachema s.r.o.
Karásek 2219/1d, 621 00 Brno
DIČ: CZ26918846 -7-

Brno, 3rd September 2025

Ref. No.AN_44_2025

