

ADVISORY NOTICE

55_2025_ New SW version for LAURA – 1.20 and extension of accepted values for glucose in Urinorm

Dear customer,

We hereby inform you that the LAURA urine reader software has been updated to SW version 1.20.

This SW update was issued to extend the accepted values for glucose (GLU) in Urinorm P to include the 100 mg/dL result during the QC measurement. Based on our internal testing, the newly accepted range for GLU in Urinorm is 100 – 1000 mg/dL / 5.5 – 55 mmol/L / 2+ – 4+. The instructions for use (IFU) for Urinorm are being updated accordingly. Please refer to this notice for the accepted values until the revised IFU are available.

The latest version of the SW is available on the [ERBA Knowledge Base](#) website.

Applied for:


INSTRUMENT:
REAGENT:

LAURA
URINORM

Cat. No: INS00063
Cat. No. REG00053

In case you have any additional questions, please do not hesitate to contact us.

Best regards,

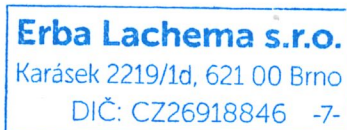

Erba Lachema s.r.o.

Brno, 21st October 2025

Ref. No. AN_55_2025