

ADVISORY NOTICE

48_2025_ New SW version for LAURA SMART – 3.1Ma (2.38) and extension of accepted values for glucose in Urinorm

Dear customer,

We hereby inform you that the LAURA SMART urine reader software has been updated to SW version 3.1 Ma (2.38).

This SW update was issued to extend the accepted values for glucose (GLU) in Urinorm P to include the 100 mg/dL result during the QC measurement. Based on our internal testing, the newly accepted range for GLU in Urinorm is 100 – 1000 mg/dL / 5.5 – 55 mmol/L / 2+ – 4+. The instructions for use (IFU) for Urinorm are being updated accordingly. Please refer to this notice for the accepted values until the revised IFU are available.

The new SW also allows you to enter the Patient ID using a barcode reader when operating in SMART Timing mode. To use this feature, please follow the steps below:

1. Wait for the green button to appear in the SMART Timing menu.
2. Scan the patient code using a barcode reader.
3. Briefly dip the strip into the patient's urine, wipe off the excess urine, and place the strip in the designated position on the pad for incubation.
4. Start the timer by pressing the green button in the SMART Timing menu. The button will turn yellow, and the Patient ID (or its 8 last characters) will appear below the corresponding button.
5. Insert the incubated strip into the strip holder tray when the device beeps and color of the button changes from yellow to orange.
6. Remove the strip from the holder tray after the measurement is complete.
7. The test result will include the patient ID when printed or exported.
8. Repeat steps 1-7 to measure the next patient sample.

The correct procedure for using SMART Timing mode with patient ID entry will also be described in the user manual. Until then, please follow the instructions provided in this notice. Please note that the provided instructions must be followed carefully to ensure safe use of the Smart Timing function with patient ID entry.

The latest version of the SW is available on the [ERBA Knowledge Base](#) website.

Applied for:

INSTRUMENT:
REAGENT:

LAURA SMART
URINORM

Cat. No: INS00064
Cat. No. REG00053



Erba Lachema s.r.o.



In case you have any additional questions, please do not hesitate to contact us.

Best regards,

Erba Lachema s.r.o.

Erba Lachema s.r.o.
Karásek 2219/1d, 621 00 Brno
DIČ: CZ26918846 -7-

A handwritten signature in blue ink, appearing to read "Petlich".

Brno, 21th November 2025

Ref. No.AN_48_2025



Erba Lachema s.r.o., Karásek 2219/1d, 621 00 Brno, Czech Republic
Identification number: 269 18 846. Tax identification number: CZ26918846
Incorporated in the Commercial Register maintained by the Regional Court in Brno, Section C, insert 45458
Tel.: +420 517 077 111, e-mail: diagnostics@erba.com, www.erba.com