

ADVISORY NOTICE

60_2025_H360_new software version SW A16.1

Dear customers,

We hereby inform you that the H360 device software has been updated to SW version SW A16.1.

1. Purpose: New items

2. List of changes in Application Software ver. A16.1:

1) Setting interface – “counting cleanser soaking”

The administrator can set enable auto cleanser soak based on the cumulative number of samples tested. When the number is reached, the system will perform an auto cleanser soak. The acceptable value ranges from 5 – 50. When the system reminds the user to perform cleanser soak, if the user cancels the operation, the system will display the reminder again after 10 minutes and it can no longer be canceled.

2) Data migration

SW version A16.1 is already included in devices produced from SN K10012539389.

For SW version A16.1, please contact your local service engineer or distributor. Please contact your local Erba representative to make sure you have enough Elite H Clean (HEM00023) in stock in order to maintain normal instrument workflow.

The latest version of the SW is available on the [ERBA Knowledge Base](#) website.

Applied for:

INSTRUMENTS:

50004676	INS00077	H360
50005448	INS00077	H360

In case you have any additional questions, please do not hesitate to contact us.

Best regards,
Erba Lachema s.r.o.

Erba Lachema s.r.o.

Karásek 2219/1d, 621 00 Brno

DIČ: CZ26918846 -7-



Brno, 18th November 2025

Ref. No.AN_60_2025

