

ADVISORY NOTICE

61_2025_XL 1000 New software update SW 2025.03A

Dear customer,

We hereby inform you that the XL 1000 device software has been updated to SW version 2025.03A.

1. Purpose:

Application software ver.2025.03A released to include new features, improvements and to address issues reported in previous application software ver.2024.08

2. List of changes in Application Software ver. 2025.03A:

Patient Reports - Translation Change: Caption is changed to Area
 Re-introduce option Use Test-wise Calibration in Calibration Policy Settings.
 When Bottle/LOT Calibration is NOT available for Reagent Bottle on-board and Use Test-wise Calibration option is selected in Settings; apply Latest Test Calibration (irrespective of same/different Reagent LOT).
 New feature: When multiple barcoded bottles of same expiry are on-board; system will use the bottles as per FIFO.
 New feature: Result Reprint has User Selection for Order to show data (Print/Display Sequence).
 Address issue: Shows incorrect required volume information for auto wash

SW version **2025.03A** is already included in devices produced from SN 115P009.

For SW versions, please contact your local service engineer or distributor.
 The latest version of the SW is available on the [ERBA Knowledge Base](#) website.

Applied for:

INSTRUMENTS:

50006796	INS00009	XL-1000 (with ISE)
50006985	INS00010	XL-1000 (without ISE)

In case you have any additional questions, please do not hesitate to contact us.

Best regards,
 Erba Lachema s.r.o.

Erba Lachema s.r.o.
 Karásek 2219/1d, 621 00 Brno
 DIČ: CZ26918846 -7-

Brno, 24th November 2025

Ref. No.AN_61_2025