

ADVISORY NOTICE

09_2026_Elite 580_ New software version SW A11.6

Dear customers,

We hereby inform you that the Elite 580 device software has been updated to SW version SW A11.6.

1. Purpose: New items/Bug fix

2. List of changes in Application Software ver. A11.6:

New items

- 1) Sarstedt tube support
- 2) Print L-J QC graph by time period
 - QC graph adds printing by time period
 - QC List Added Print by Time Period
- 3) Cleanser soaking logic optimization- the counting cleanser soaking can only be cancelled once. If the counting cleanser soaking fails due to a malfunction, the accumulated sample count will be reset to zero. Once the prompt box for counting cleanser soaking appears, the process will be interrupted due to power failure, and the accumulated sample count will not be reset. When reminding me again, forced soaking cannot be canceled.

Bug fix

- 1) BUG fix#57710 - Horizontal motor position error
- 2) BUG fix#63779 - Open mode testing with higher PLT results
- 3) BUG fix #60209 - Normal testing results show all-star marks

Based on long-term data collection and performance monitoring, we strongly recommend upgrading the software to ensure the instrument maintains its optimal performance characteristics throughout its entire lifecycle. Implementing this update helps safeguard consistent results and supports long-term operational efficiency.

SW version A11.6 is already included in devices produced from SN K11052549001.

For SW version A11.6, please contact your local service engineer or distributor.

The latest version of the SW is available on the [ERBA Knowledge Base](#) website.

Applied for:

INSTRUMENTS:

50004675	INS00071	Elite 580
50004873	INS00071	Elite 580

In case you have any additional questions, please do not hesitate to contact us.

Best regards,
Erba Lachema s.r.o.

Erba Lachema s.r.o.
Karásek 2219/1d, 621 00 Brno
DIČ: CZ26918846 -7-

Brno, 26th February 2026

Ref. No.AN_09_2026

