

ADVISORY NOTICE

17_2026_CHEM-7_New embedded software version v2025.C0.04

Dear customer,

We hereby inform you that the CHEM-7 device embedded software has been updated to SW version 2025.C0.04.

1. Purpose:

Embedded software ver. 2025.C0.04 for CHEM7 is released to include new feature and to address issue reported in previous embedded software ver.2025.C0.03.

2. List of changes in Embedded Software ver. 2025.C0.04:

1. New feature:

Implementation of MSD (Mass Storage Device) I/E (Import/Export) feature

- New field added for test revision number in parameters for all tests. User allowed editing revision number
- (From 0 to 9 per year).
- MSD I/E (Import/Export) menu added in System Parameters -> System Setup -> Self Test menu.
- In Import Erba Tests, a. Import New Test, b. Import Selected Test, c. Import All Test options added.
- In Export Erba Test, a. Export Selected Erba Test & b. Export All Test options added.
- Print Test Revision Number (on Thermal Printer). submenu added in Self Test menu
- New field added for test revision number in test parameters for all tests.
- (Note that tests revision number gets printed in test parameters & run tests print (wherever possible) & not all places)

Firmware has been modified for tests having "RATE A" (only) mode for following changes

- All digits are considered instead of 3 after decimal places for Net Absorbance & Result Calculation.
 - Round Up effect has been disabled.
2. For "uALB" test default temperature changed from 25°C to 37°C.
 3. Bug fixed: Addressed the issue of "Unexpected memory reset (through downloading parameters) due to memory corruption happens at power off-on the unit sometimes".

Embedded SW version **2025.C0.04** is already included in devices produced from SN S4142 and S4146 onwards.

For SW versions, please contact your local service engineer or distributor.
The latest version of the SW is available on the [ERBA Knowledge Base](#) website.

Applied for:

INSTRUMENTS:

52000103	INS00014	CHEM-7 - ECZ
50005172	INS00014	CHEM-7
50008058	INS00014	CHEM-7 WITH INCUBATOR

In case you have any additional questions, please do not hesitate to contact us.

Best regards,
Erba Lachema s.r.o.

Erba Lachema s.r.o.
Karásek 2219/1d, 621 00 Brno
DIČ: CZ26918846 -7-

Brno, 16th April 2026

Ref. No.AN_17_2026

