

ADVISORY NOTICE

22_2026_XL-180 new embedded software ver.XL180_2025.RE.01

Dear customer,

We hereby inform you that the XL180 embedded software has been updated to ESW ver. XL180_2025.RE.01

1. Purpose:

Embedded software ver.XL180_2025.RE.01 released to address issue reported in previous Embedded software ver. XL180_2024.RE.01.

2. List of changes in Embedded Software ver. XL180_2025.RE.01:

Issue Resolved - volume Scan during Run/Pre run - SRGT Rotational error during R1 operation. Spartan Board was not able to capture this error & this error will now be reported only when the tray movement is for more than 800 microsteps, this is done so that the pos opto alignment is not critical

Embedded SW version **XL180_2025.RE.01** is already included in devices produced from SN S170042, S170043, S170045, S170055 onwards.

For SW versions, please contact your local service engineer or distributor.
The latest version of the SW is available on the [ERBA Knowledge Base](#) website.



Applied for:

INSTRUMENTS:

50006793	INS00037	XL-180 (with ISE)
50006792	INS00038	XL-180 (without ISE)

In case you have any additional questions, please do not hesitate to contact us.

Best regards,
Erba Lachema s.r.o.

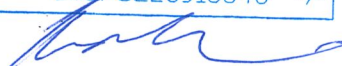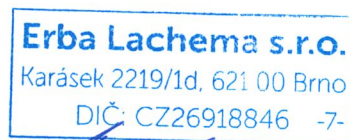

Brno, 15th July 2026

Ref. No.AN_22_2026