

ADVISORY NOTICE

28_2026_Total Bilirubin, Sample Type

Products: Bilirubin Total DCA, BIL T DCA 330
Ref. Code: XSYS0087

Dear customers,

We want to inform you regarding to change of sample type acceptable to Bilirubin Total DCA assay. Acceptable specimens are:

- Serum
- K₂-EDTA plasma

With the previously reported Li-heparinized plasma, it was not possible to ensure a non-interfering Li-heparin concentration of up to 5 IU/ml in all types of commercially available collection tubes. For this reason, this sample type has been removed from the instructions for use.

Please update your instruction for use.

In case of any questions, please do not hesitate to contact your local ERBA team.

Best regards,

Erba Lachema s.r.o.
Technical Product Management



Brno, 31st August 2026

Ref. No.AN_28_2026