

ADVISORY NOTICE

31_2026_ECL 760_ new software version sw 1.3fe

Dear customers,

We hereby inform you that the ECL 760 device software has been updated to SW version 1.3fe.

1. Purpose: To introduce improvements to the previous software version

2. List of changes in Application Software ver. 1.3fe:

- 1) Updated the default post-flush settings for the TT and FIB assays. The cleaning solution for both the pre-flush and post-flush is now required only in position A1.
- 2) Changed the AE threshold for D-Dimer FEU and DDU from 0.03 to 0.05.
- 3) Added support for 0/1 dilution factor.
- 4) Modified the sample testing interface to automatically remove sample request information on the following day.
- 5) Added a CLEAN/PERFUSION button to the System interface.

SW version 1.3fe is already included in devices produced from SN 0551-71-260804.

This software version is applicable to the new ECL 760 version, starting from SN 0478-71-030725.

For SW version 1.3fe, please contact your local service engineer or distributor. The latest version of the SW is available on the [ERBA Knowledge Base](#) website.


Applied for:

INSTRUMENTS:

10020973	INS00070	ECL 760
----------	----------	---------

In case you have any additional questions, please do not hesitate to contact us.

Best regards,
Erba Lachema s.r.o.

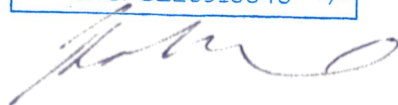

Brno, 25th September 2026

Ref. No.AN_31_2026